**IKP - IAP - ESP**

**CONICAL CONNECTION 12°**

# company profile

**RESISTA**<sup>®</sup>

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**RESISTA**, trademark owned by Ing. Carlo Alberto Issoglio & C. S.r.l. , identifies the full range of products designed by the company to meet the needs of dentists and dental technicians.

**RESISTA is known all over the world since 1946, and synonymous of guarantee, balance and continuity.**

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The company has one of the most ip-to-date manufacturing site for the medical devices, located in Omegna, a beautiful town on Orta Lake, Italy.

With commitment, merit and pride, the technical and scientific staff of the R&D department is daily involved to promote innovation and the improvement of our products and processes.

Over the years, the **RESISTA** brand has also become synonymous of fast and qualitative response to market changes and the needs of modern dentistry.

Quality, Price and Service summarize in three simple words the daily perception of the users of **RESISTA** branded products.

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**RESEARCH & DEVELOPMENT** - The R&D Resista Group's program stems from indications from the clinical world combined with our experience in the field of implantable medical devices.

Due to the use of 3D modeling programs and sophisticated computer systems, we simulate the final geometries and the design, developing the rapid prototyping phases of the projects.

**INNOVATIONS** - Resista Group is structured to meet all the needs of the metal-implant with an advanced conception technology, prototyping and final product engineering to arrive at innovative solutions. Validation is carried out in collaboration with accredited Research Institutes following the International Standards Requirements.

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**PROJECT AND PROCESS QUALITY** - ICIM Spa certified **Ing. C. A. Issoglio & C. S.r.l.** in accordance with **UNI EN ISO 9001** and **UNI EN ISO 13485** in compliance with all current regulations relating to the products and services offered. Every employee covering different roles (technical, engineering, administrative, commercial, mechanic operators, etc.), follows guidelines and objectives for a single purpose: continuous product improvement.

**MAIN OBJECTIVE** - Customer satisfaction is our objective. The winning factor is the company's ability to quickly resolve any type of request. The technical support provided by Product Specialists, availability, efficiency and friendliness are the strengths of our structure.

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**BIOMECHANICAL CONCEPTS** - The **Resista Group** method success is the result of several product design studies and the high technology applied in the production. The precision and reliability of the implant has a strong influence on the long term clinical success.

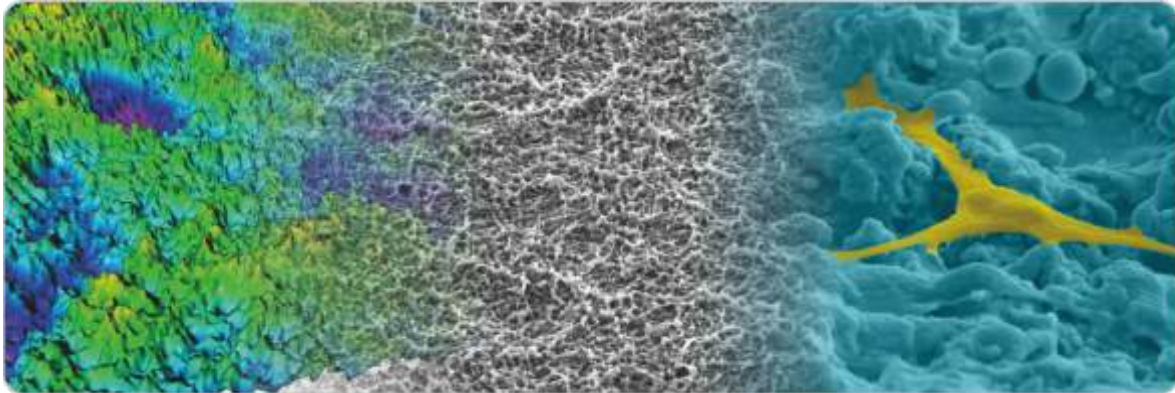
**The manufacturing process** is carried out by high qualified operators, skilled enough to obtain optimal and reproducible results with the use of electronically controlled CNC machinery. The machinery tolerances, especially in **prosthetic connections**, are our **gold standard** and are between **7 - 10  $\mu\text{m}$** .

**MATERIALS** - The implants are made of Titanium ASTM Gr. 4 (ISO Standard 5832/2) and the screws and prosthetic components are made of Titanium ASTM Gr. 5 (ISO Standard 5832/3).

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
**SURFACE TREATMENTS** - The non-contaminant micro-subtraction process, **DAE** (Double Acid Etching), changes the implant micro roughness (**Ra, Rq**) and the surface texture maximizing the wettability and biomimetic properties. The new surface treatment and decontamination process are patented by **Nobil Bio Ricerche** improving the bio-chemical properties of the implants.





**QUALITY SYSTEM (QS)** - **UNI EN ISO 9001** and **UNI EN ISO 13485** in compliance of Medical Devices Directives. We are qualified in the design and in the production management of dental implants, dental prosthesis, intraligamental anesthesia syringes and abrasive discs.

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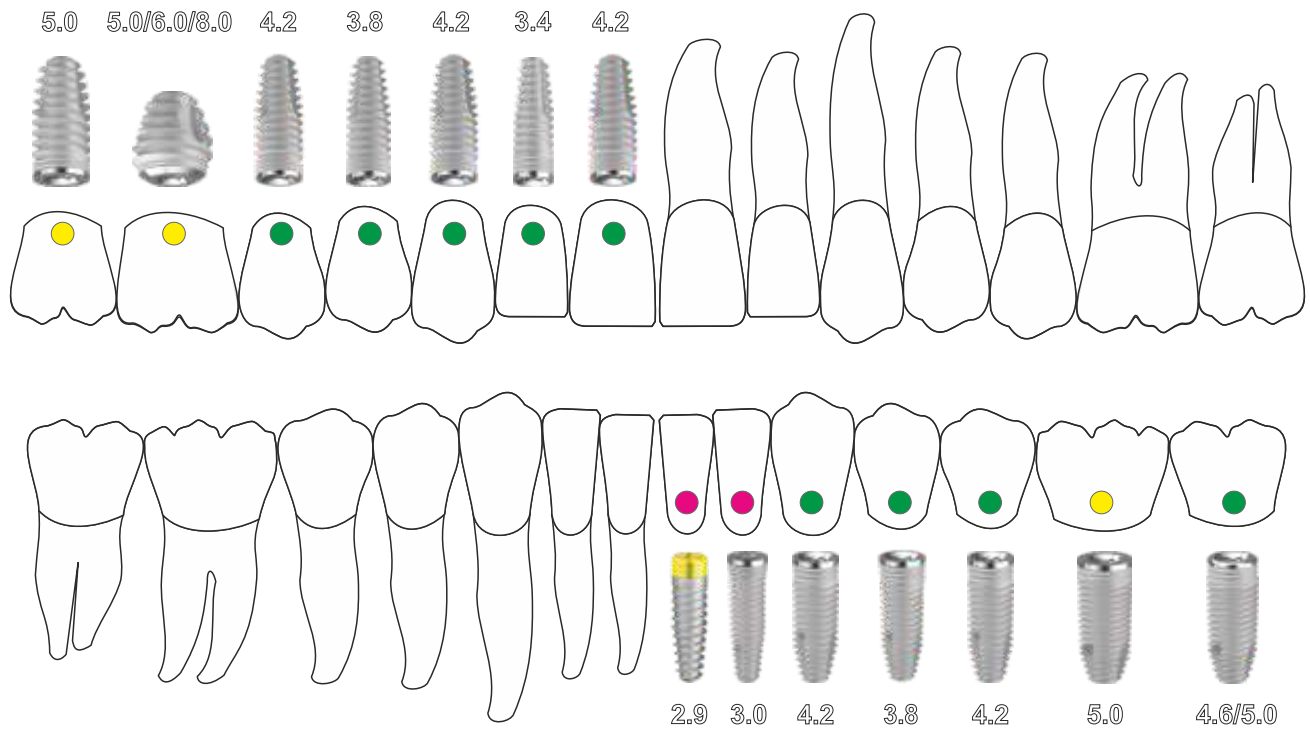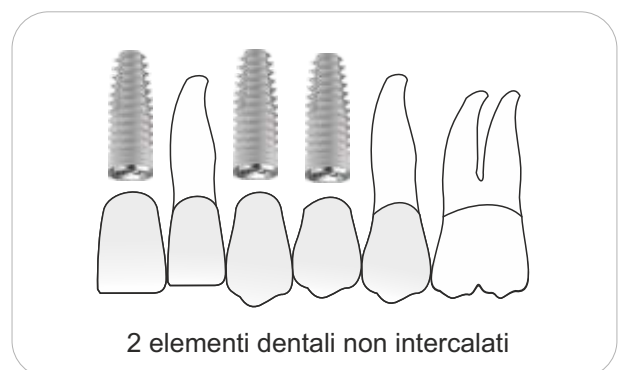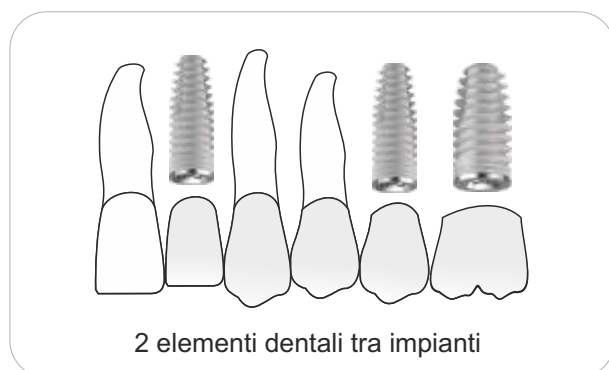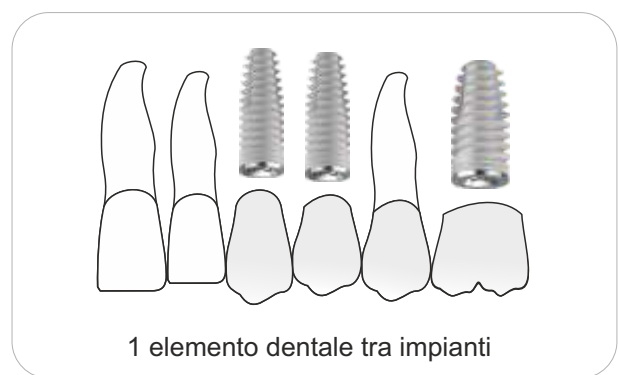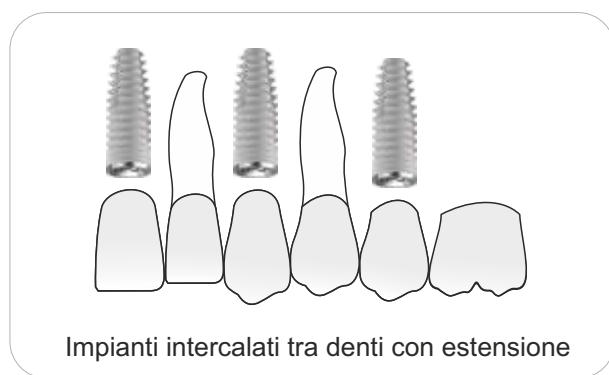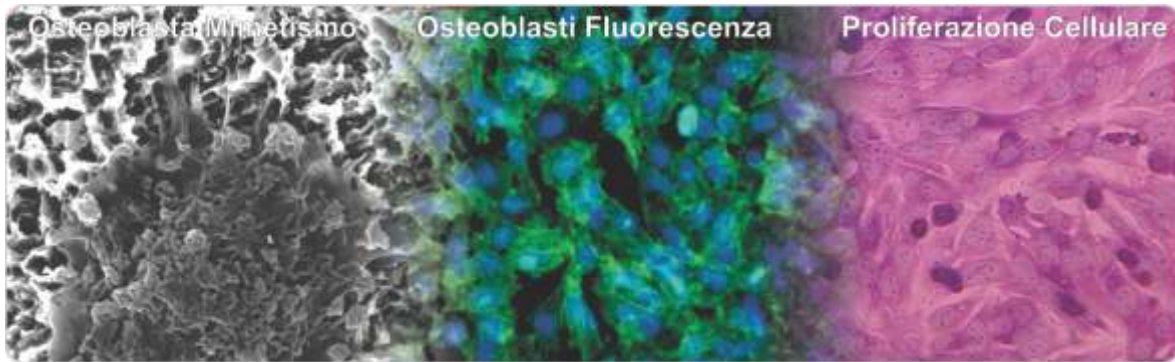
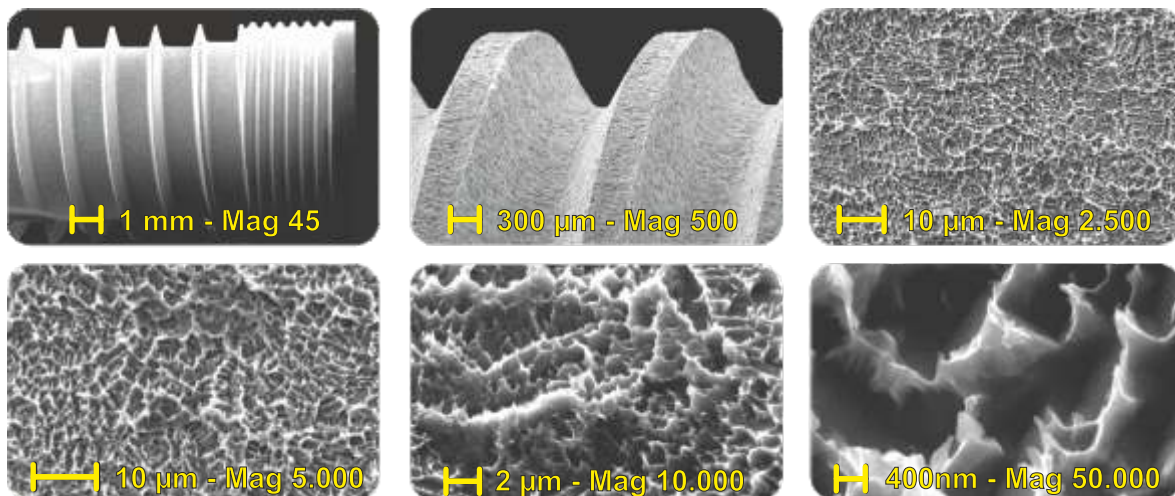
## The Micro-Nano Rough DAE surface treatment speed up bone healing processes

1. Removes the manufacturing organic residuals along with Argon Plasma Cleaning
2. Smooths the edge and eliminates the micro-defects
3. Increases surface and wettability (hydrophilicity surface), improving the first fibrin bridges adhesion
4. Increases protein adhesion
5. Maximizes cellular adhesion through the Micro-Nano roughness suitable for actin filaments (phyllopods) anchoring
6. Changes the Titanium surface chemical characteristics improving the biocompatibility and increasing the cellular proliferation and vitality

## Surface topography and chemistry interact with the cellular differentiation processes

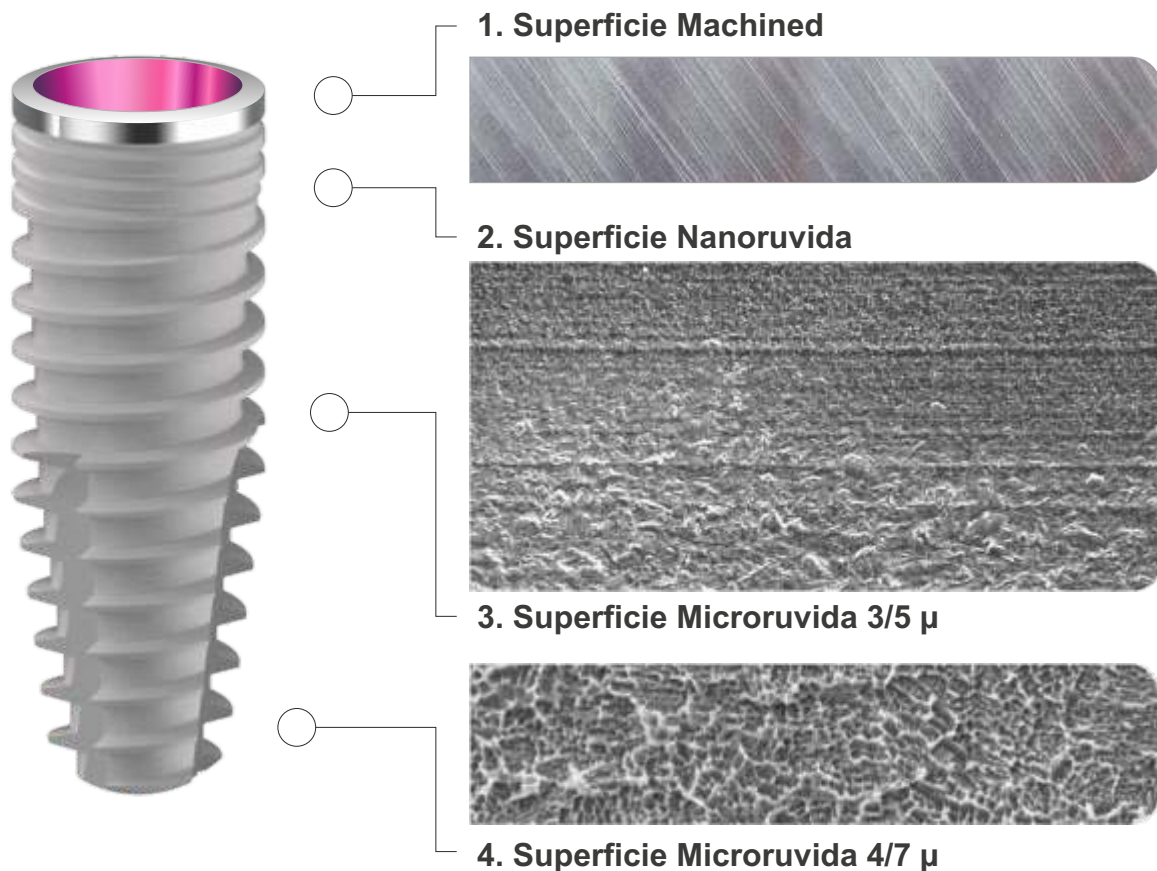
The implant surface uses the microtopography as a communication language with the host tissue cells. The Plasma Argon Cold Cleaning with a controlled-environment packaging at a strict protocol procedures, play a key-role in the control of adhered endotoxins (main immunological response player to implant surfaces).

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### VEGF/VEGF-R/RUNX2 Upregulation in Human Periodontal Ligament Stem Cells Seeded on Dual Acid Etched Titanium Disk

Francesca Diomede, Guya Diletta Marconi, Marcos F. X. B. Cavalcanti, Jacopo Pizzicannella, Sante Donato Pierdomenico, Luigia Fonticoli, Adriano Piattelli and Oriana Trubiani

Materials 2020, 13, 706; doi:10.3390/ma13030706

## SURFACE TREATMENT DISTRIBUTION ON IMPLANT'S BODY

All the **Resista's** implants have a surface treatment that is locally differentiated. The logic of surface treatment distribution is dictated by the endo-osseous implant position which requires different characteristics dependin on the muco-gingival component presence around.

- 1. Machined Surface:** Brushed, rectify and protected during the roughening process
- 2. Nano-rough Surface:** Speedy treatment, nano-rough with average Ra less than one micron
- 3. Micro-rough Surface: 3/5 $\mu$ :** DAE Treatment, micro-rough with average Ra between 3/5 $\mu$
- 4. Micro-rough Surface: 4/7 $\mu$ :** DAE Slow Treatment, micro-rough with average Ra between 4/7 $\mu$

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### The Bacterial Anti-Adhesive Activity of Double-Etched Titanium (DAE) as a Dental Implant Surface

Morena Petrini, Alessandra Giuliani, Emanuela Di Campi, Silvia Di Lodovico, Giovanna Iezzi, Adriano Piattelli and Simonetta D'Ercole<sup>1</sup>  
International Journal of Molecular Sciences 2020, 21, 8315; doi:10.3390/ijms21218315

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## THE RESISTA'S IMPLANT LINE STRONG POINTS

- 1. Macro Geometry:** A vast range of implant shapes for different surgical needs (Cylinder, Tapered, Active, Short, Mini and Extra-Large)
- 2. Micro Geometry:** Last Generation Differentiated Surface (Full, Half and Machined) treated (DAE Micro and Nano rough), decontaminated in Argon Cold Plasma Reactor.
- 3. Multifunctional Mounter:** 4 components in a single article (Mounter, Transfer, Abutment and Mounter for Computer Guided Surgery) to maximize result and minimize costs.
- 4. Prosthetic Parts:** A very wide range of prosthetic variants with 7/10 micron machining tolerances on the connection, with emerging curvilinear profiles, flat and conical connections, platform switching, golden titanium screws and PVD TiN coatings (more aesthetic and less retention of dental plaque).
- 5. Surgical Instruments:** Surgical Drills **3-Tech**, maximum efficiency, minimum trauma, smaller friction coefficient, **PRO MSD** technology for counterclockwise bone densifying application long, lasting resistance, perfect visibility of laser marking and surgical millimeters stop.

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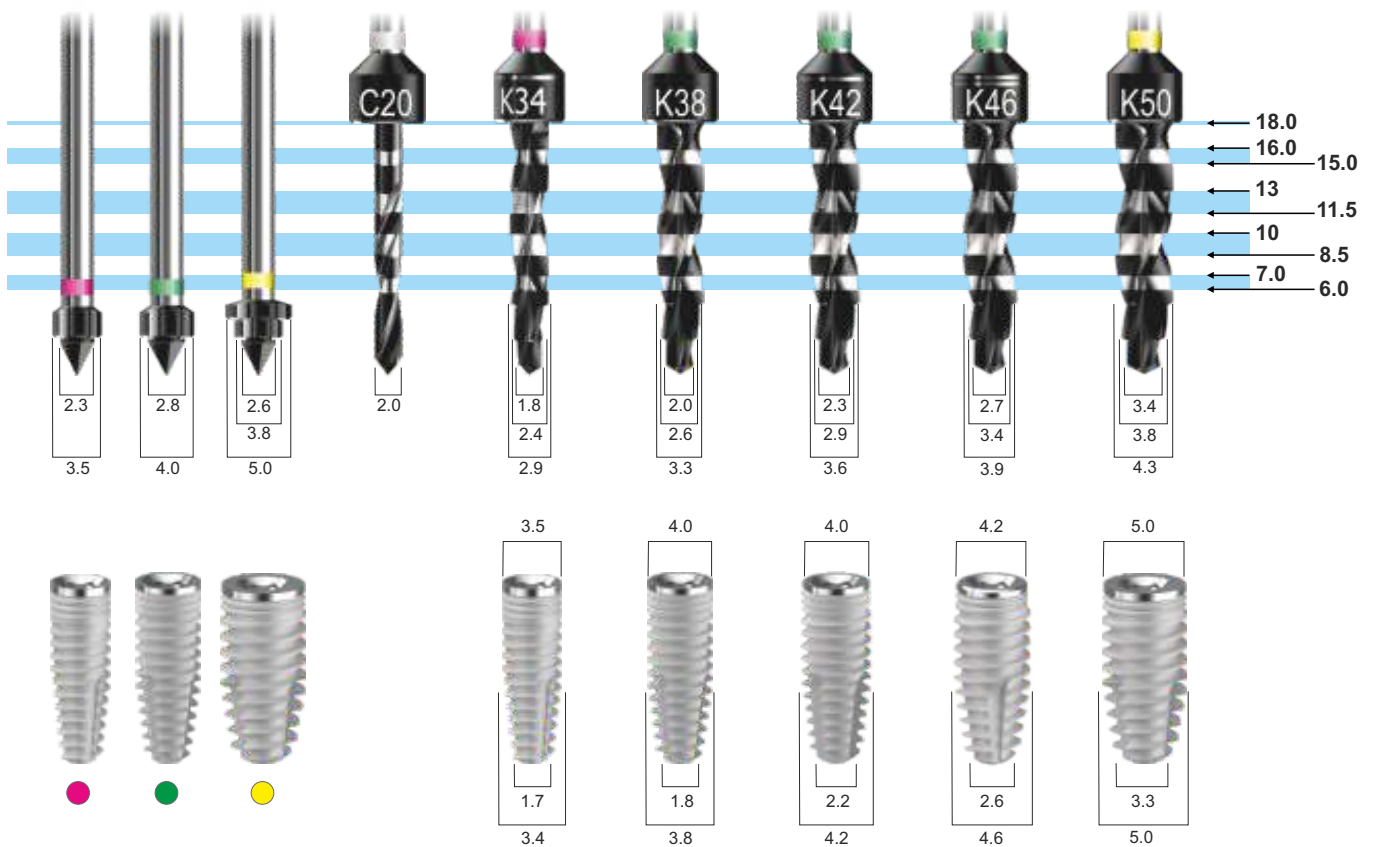
### Various bio-mechanical factors affecting heat generation during osteotomy preparation: A systematic review

Chirag J Chauhan<sup>1</sup>, Darshana N Shah<sup>1</sup>, Foram B Sutaria<sup>1</sup>

Indian J Dent Res. Jan-Feb 2018;29(1):81-92. doi: 10.4103/ijdr.IJDR\_729\_16.

## LOGIC RELATION BETWEEN DRILL AND IMPLANT CORE

The rotary instruments design and shape is based on the logical coherence between the implants size, in the external dimensions, internal core and their penetration properties.

The **IC** line **Cylindrical Implants** have a tapered apical portion 3mm long.

The **IK** line **Conical Implants** have a tapered apical portion 5mm long.

The drills have respectively 2/3 apical sections, suitable for the implant tunnel preparation, with respect to the implant geometry, maintaining the best bone implant contact (BIC).

Each step drill is equipped with a sharp cutting edge to facilitate penetration, minimizing friction and overheating.

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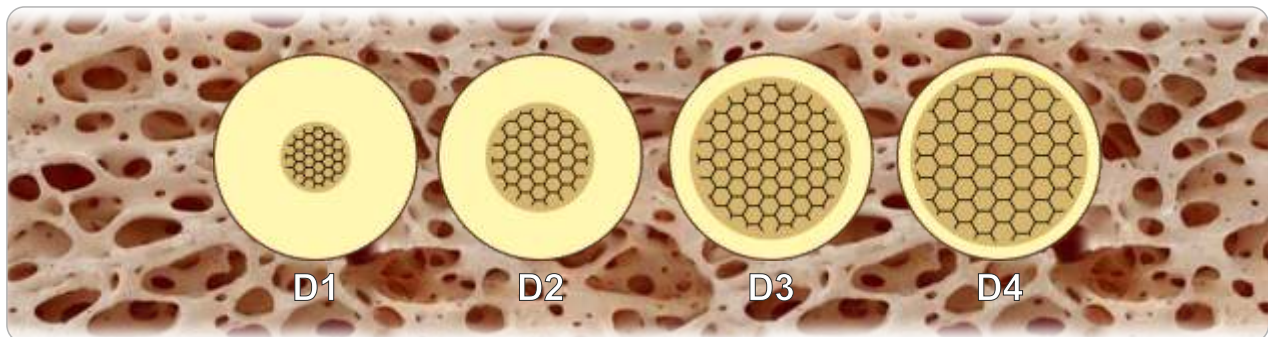
### ✓ Bone classification: clinical-histomorphometric comparison

Trisi P, Rao W. - Clin Oral Implants Res. 1999 Feb;10(1):1-7. doi: 10.1034/j.1600-0501.1999.100101.x.

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## THE CONDITIONING OF SURGICAL PROTOCOLS

The differentiated implant geometries in the macro/micro shape provide for different applications, both according to the surgical/prosthetic requirements (immediate loading, early loading or postponed loading) and according to the conditions of the bone/anatomical component of the patient who needs rehabilitation. However, surgical maneuvers and alternative protocols are often required to improve implant performance, according to the morphological characteristics of the residual bone structures.

The success of these maneuvers, that is often the result of a customized tools and protocols combination, represents the different response to the problem in relation to the personal operating skills and the presence of tools suitable for their application.

## MECHANICAL OSSEODENSIFICATION IN ANTI-CLOCKWISE ROTATION

The osseodensification using rotary instruments is a recent surgical technique for the implant site preparation that can be associated with different protocols; these protocols can be used in those particular anatomical conditions such as poor bone quality and insufficient vertical / horizontal dimensions.

This non-subtractive osteotomic bone-condensation approach produces an increase in peri-osteotomic bone density, saving the tissue and increasing the primary implant stability.

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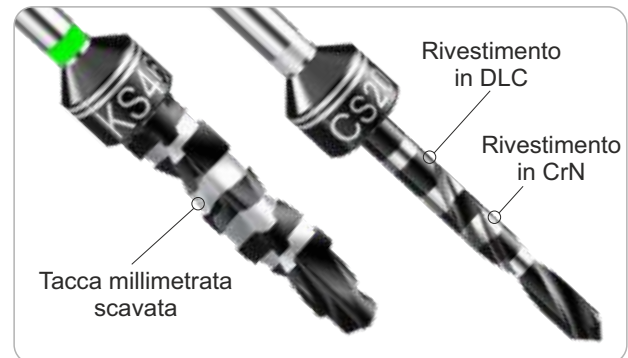
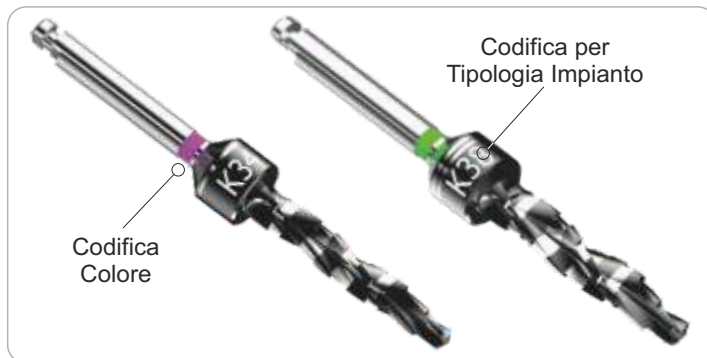
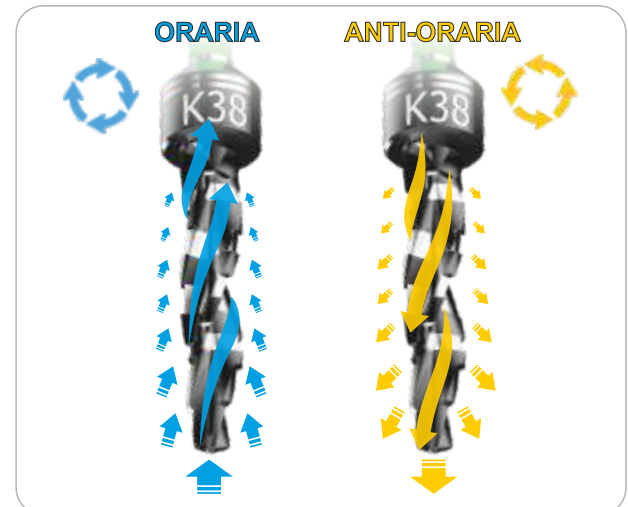
## RESISTA PRO MSD SURGICAL DRILLS - Modular Surgical Drilling

The Modular Surgical Drills with variable geometry are 3-section helicoidal drills with progressive radius rake, that can be used clockwise and anticlockwise.

The diameter of each drill changes according to a constant ratio of 0.4mm (30/34/38/42/46/50): this allows the operator to choose the drill according to the bone quality (over-preparation or under-preparation). All the drills are coated with a DLC (Diamond Like Carbon) synthetic diamond solid lubricant that maximizes performance in terms of mechanical strength and friction reduction.

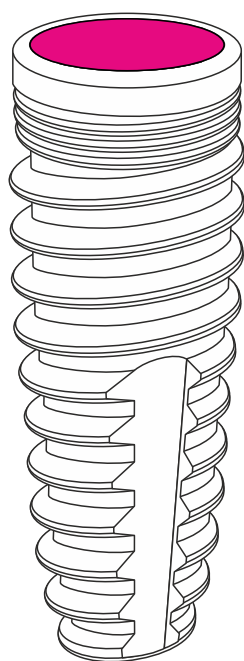
The **anticlockwise rotation**, reversing the involved forces, generates 3 different effects on the implant tunnel perforation, which can revolutionize the known milling logic.

- 1) Anterior and lateral thrust of the bone removed by the tip, and of the liquids as well, blood and physiological water.
- 2) Ejection and "hammer effect" of the drill that produce an improvement in vertical control.
- 3) Reduction of cutting efficiency to protect sensitive anatomical parts.

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

# IK 34-P

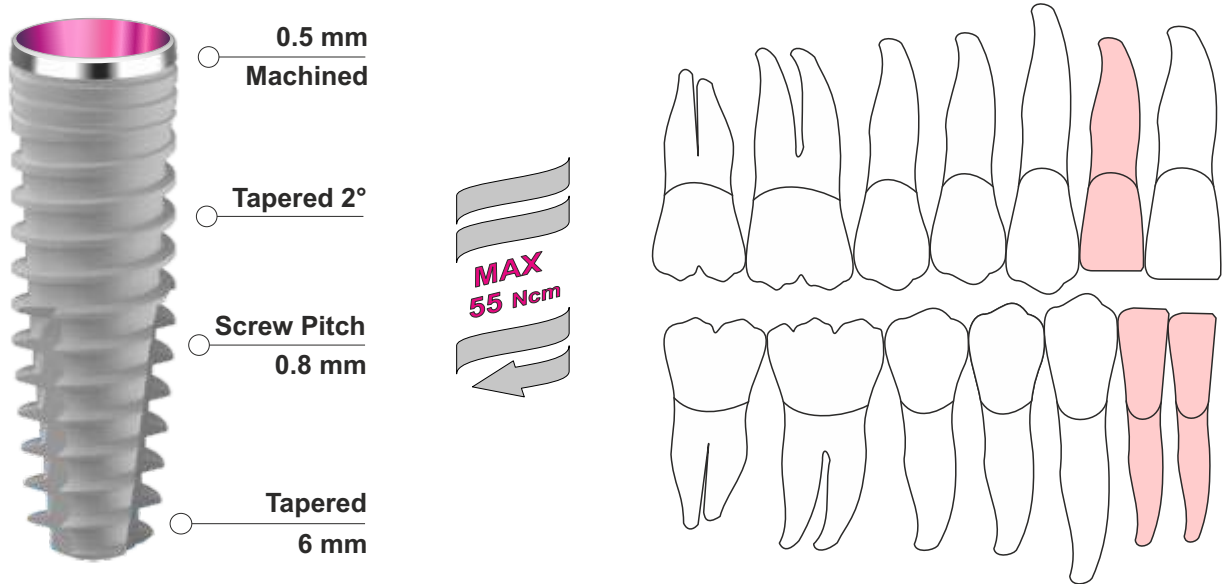**NARROW K- PLATINUM**  
conical connection 12° - full treatment

## TAPERED SHAPE MOUNTERLESS


CODE	IMPLANT	LENGTH	NECK	APEX
IK 3407 P	Ø 3.4	7.0 mm	Ø 3.5	Ø 1.7
IK 3408 P	Ø 3.4	8.5 mm	Ø 3.5	Ø 1.7
IK 3410 P	Ø 3.4	10 mm	Ø 3.5	Ø 1.7
IK 3411 P	Ø 3.4	11.5 mm	Ø 3.5	Ø 1.7
IK 3413 P	Ø 3.4	13 mm	Ø 3.5	Ø 1.7
IK 3415 P	Ø 3.4	15 mm	Ø 3.5	Ø 1.7

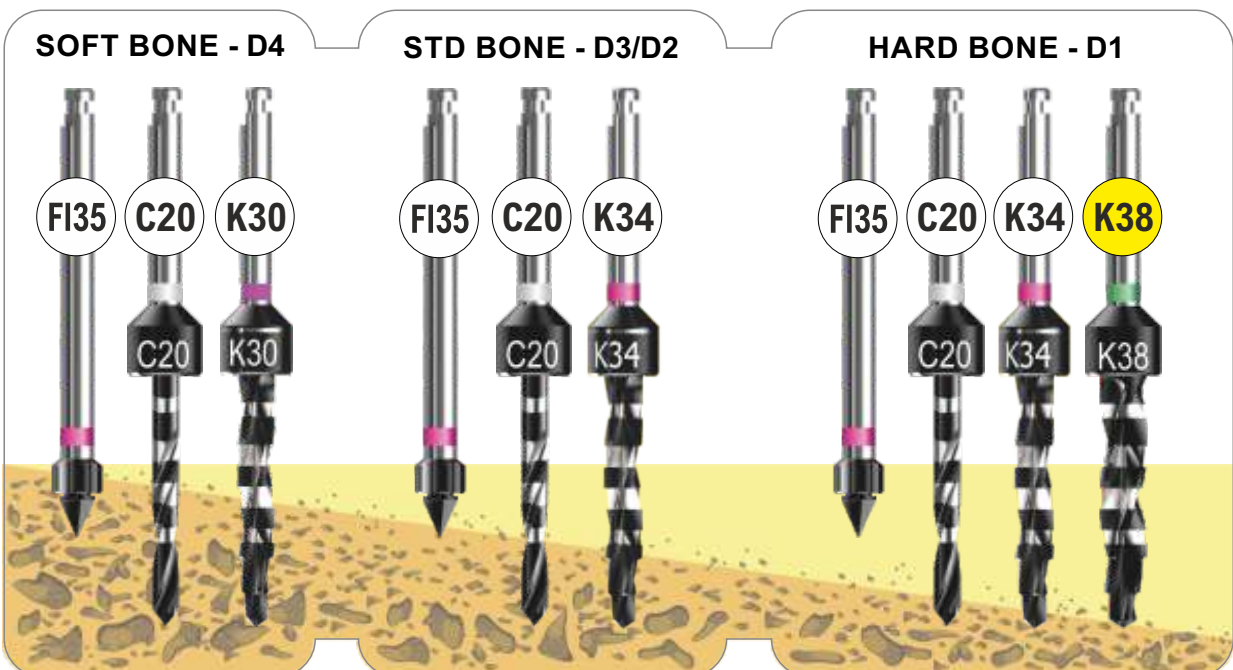
**NARROW K- PLATINUM**  
conical connection 12° - full treatment

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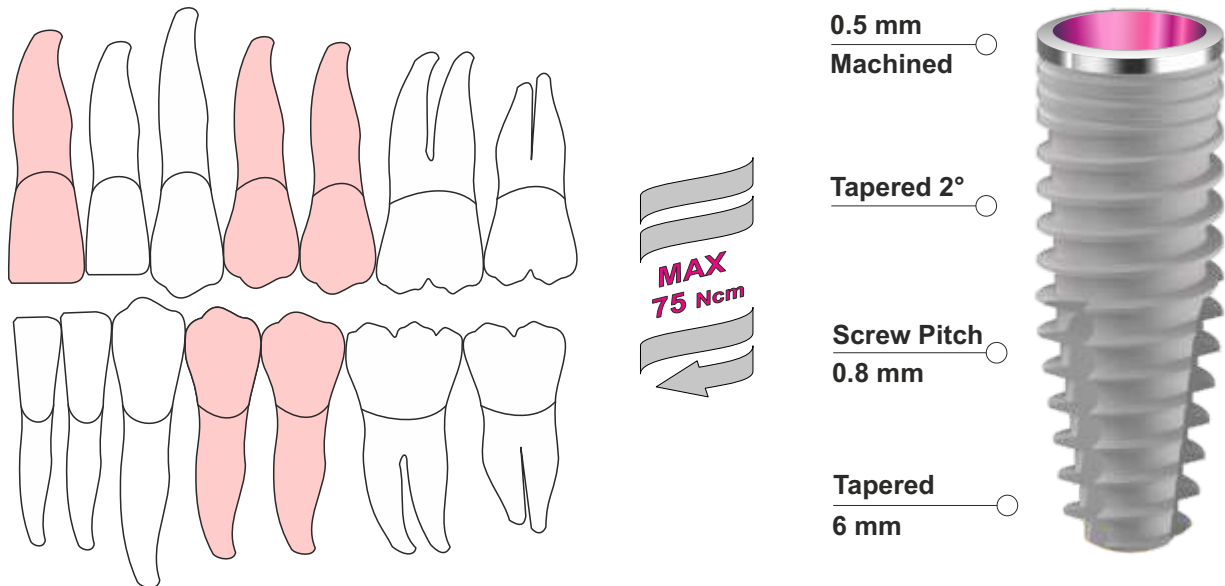
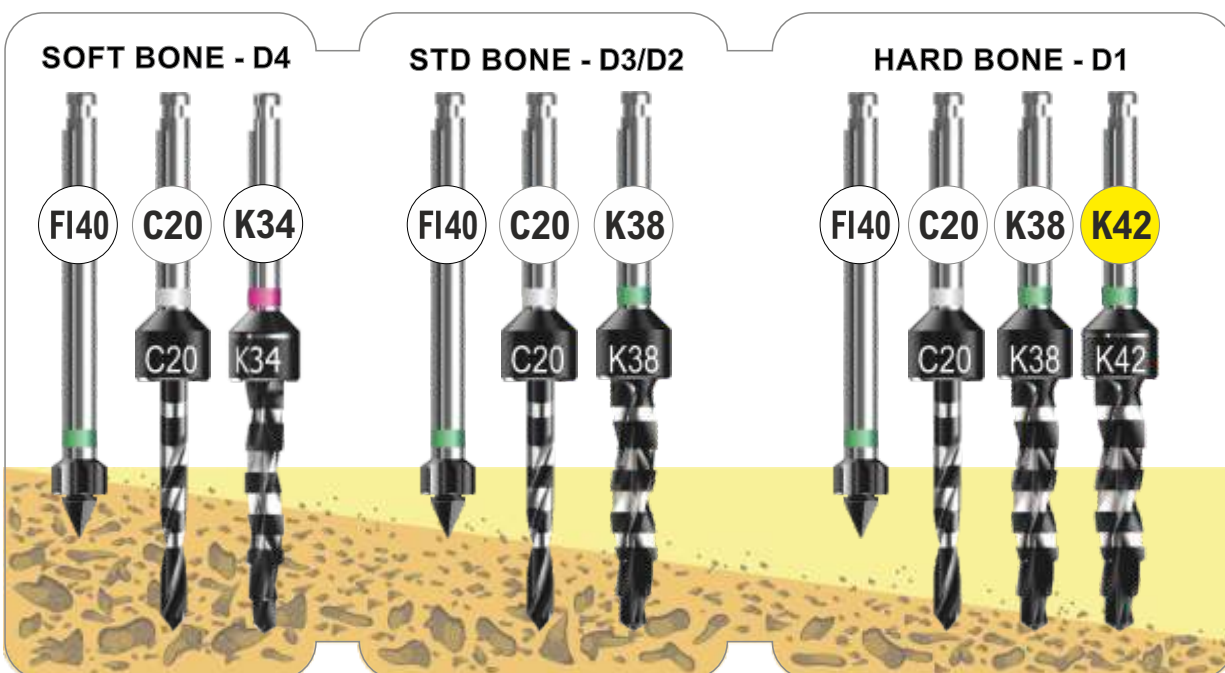
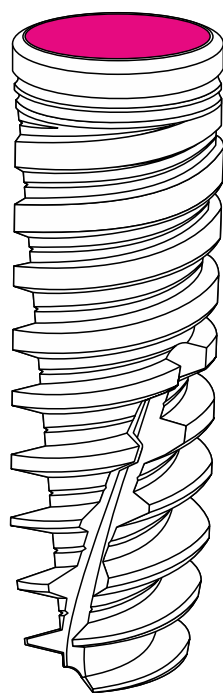
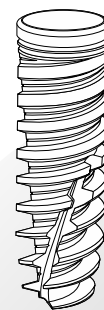
# IA 34-P

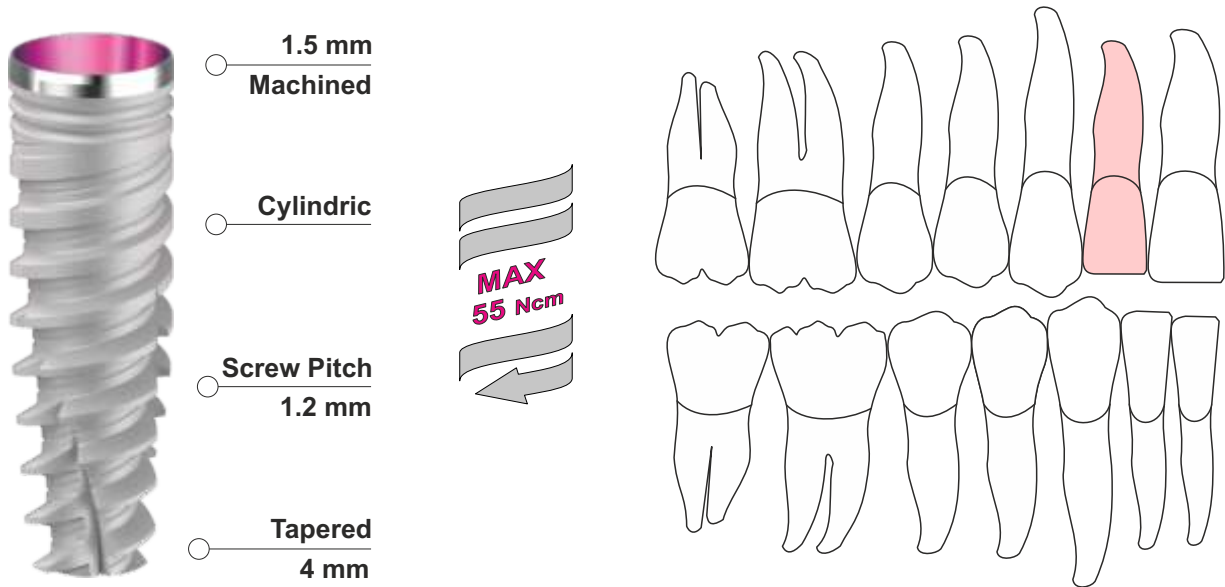**NARROW A - PLATINUM**  
conical connection 12° - full treatment

## ACTIVE BLADE SHAPE MOUNTERLESS

CODE	IMPLANT	LENGTH	NECK	APEX
IA 3408 P	Ø 3.4	8.5 mm	Ø 3.5	Ø 1.8
IA 3410 P	Ø 3.4	10 mm	Ø 3.5	Ø 1.8
IA 3411 P	Ø 3.4	11.5 mm	Ø 3.5	Ø 1.8
IA 3413 P	Ø 3.4	13 mm	Ø 3.5	Ø 1.8
IA 3415 P	Ø 3.4	15 mm	Ø 3.5	Ø 1.8

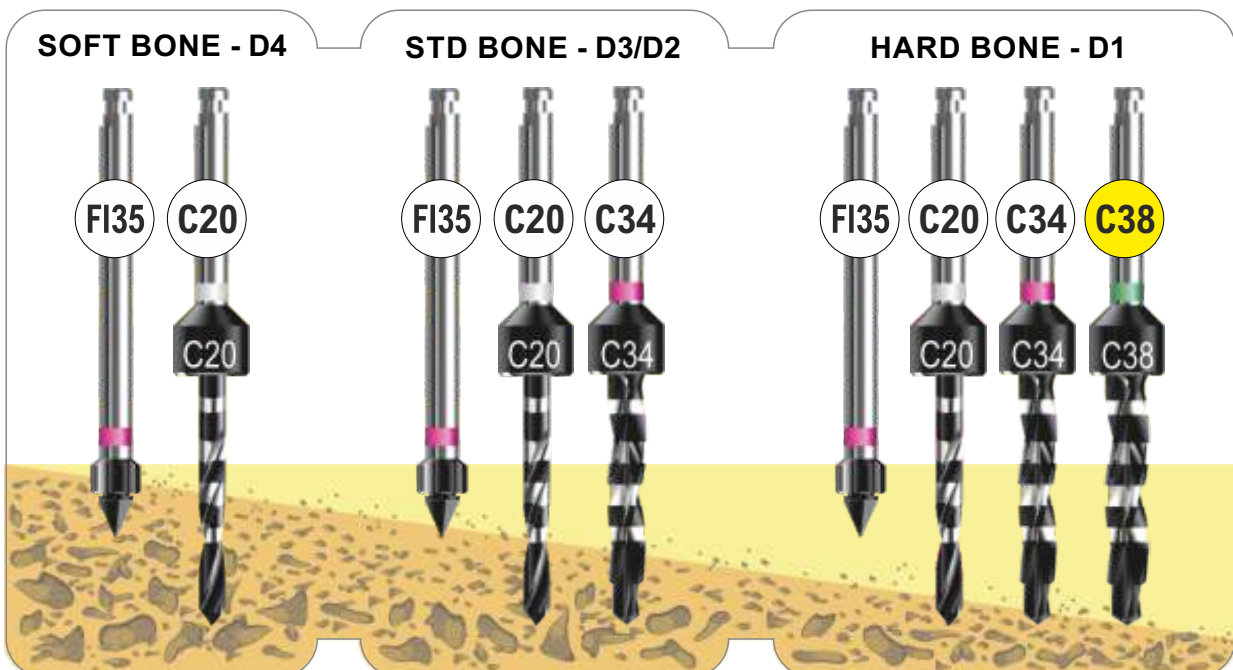
# NARROW A - PLATINUM


conical connection 12° - full treatment

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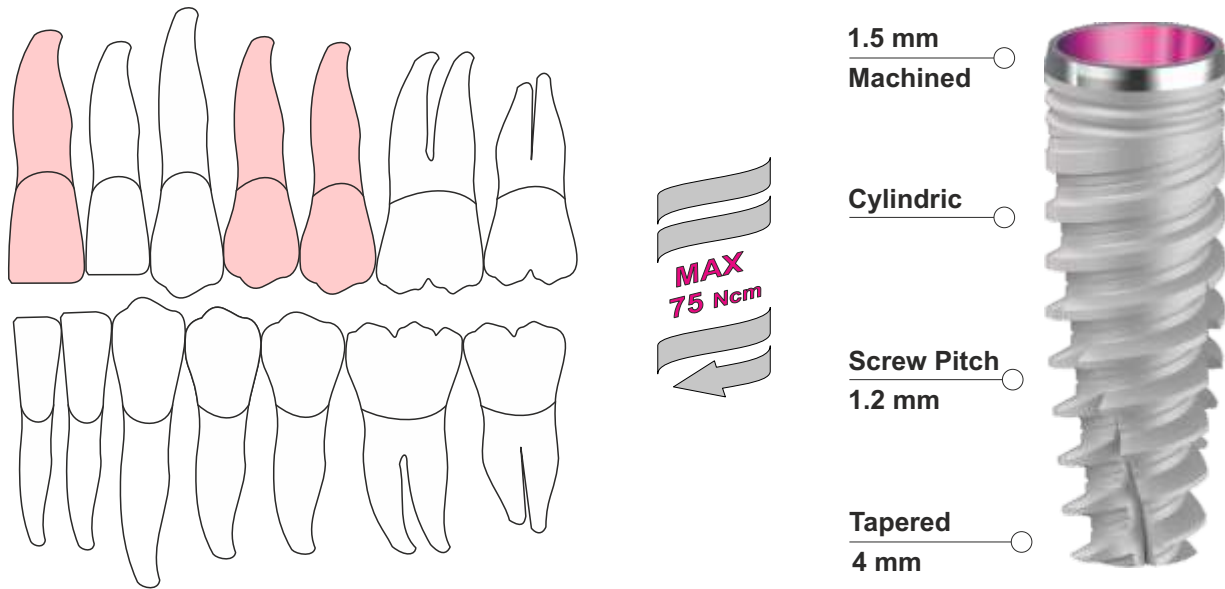
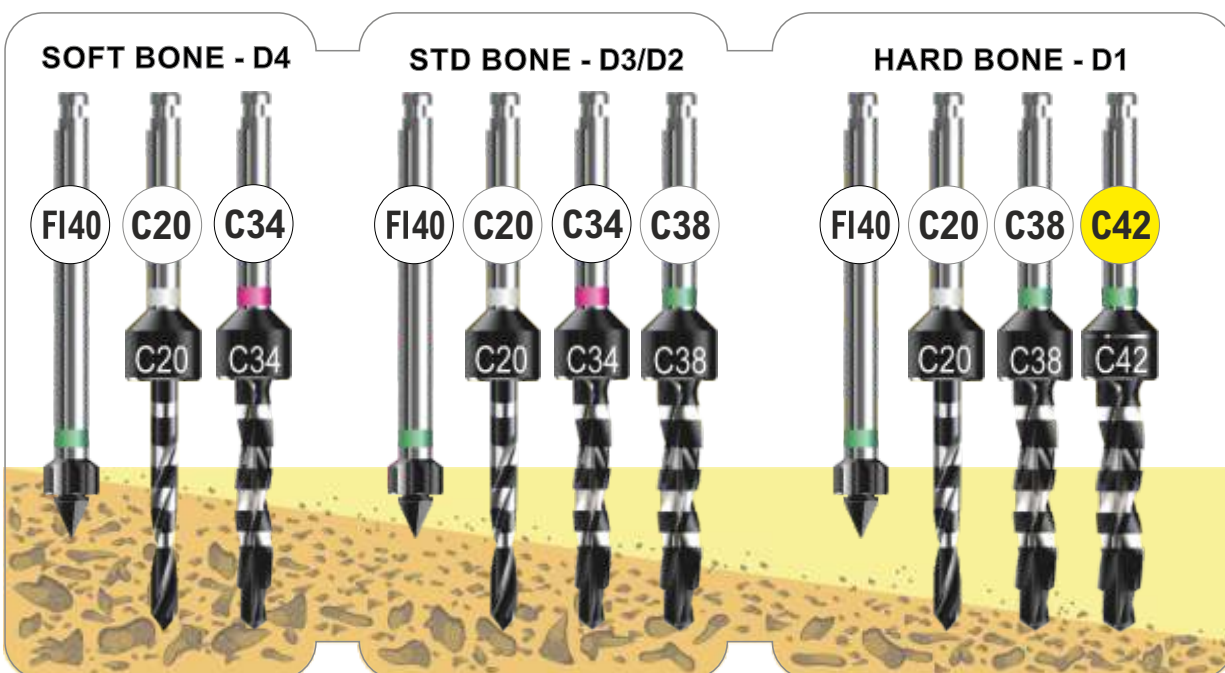
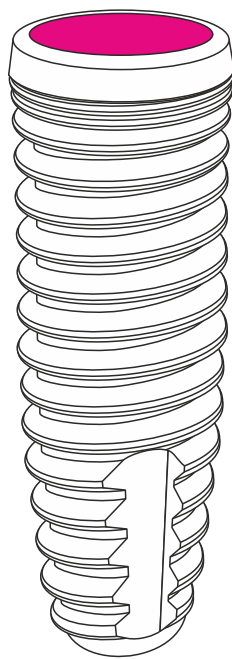
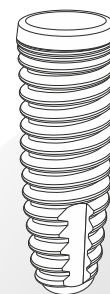
# ES 34-P

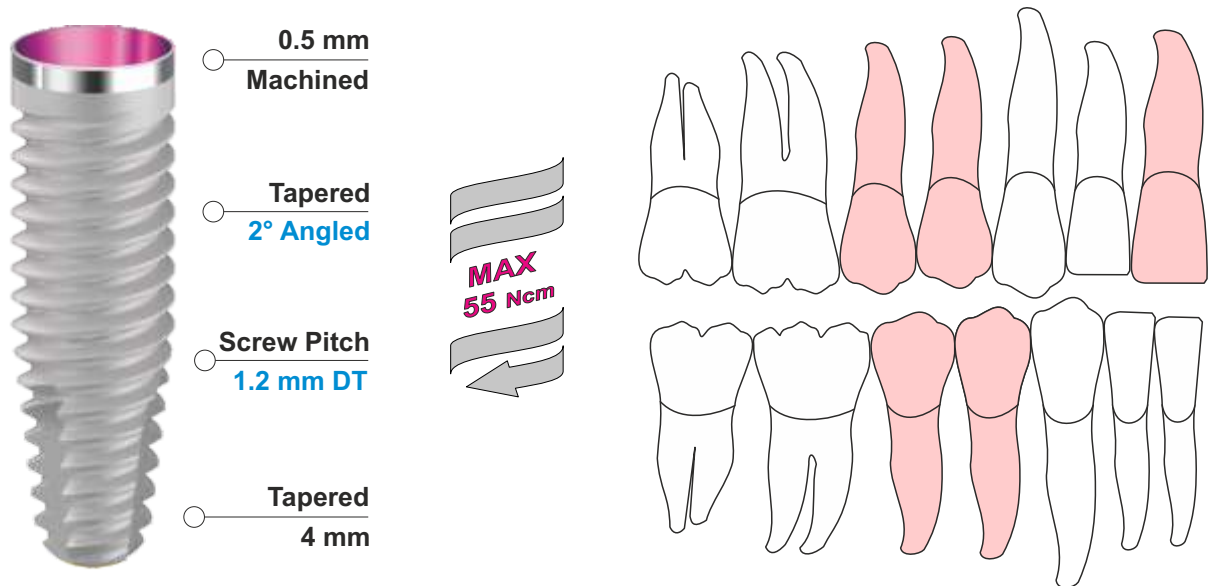**NARROW S- PLATINUM**  
conical connection 12° - full treatment

## SPEEDY SHAPE MOUNTERLESS


CODE	IMPLANT	LENGTH	NECK	APEX
ES 3408 P	Ø 3.4	8.5 mm	Ø 3.5	Ø 1.8
ES 3410 P	Ø 3.4	10 mm	Ø 3.5	Ø 1.8
ES 3411 P	Ø 3.4	11.5 mm	Ø 3.5	Ø 1.8
ES 3413 P	Ø 3.4	13 mm	Ø 3.5	Ø 1.8
ES 3415 P	Ø 3.4	15 mm	Ø 3.5	Ø 1.8

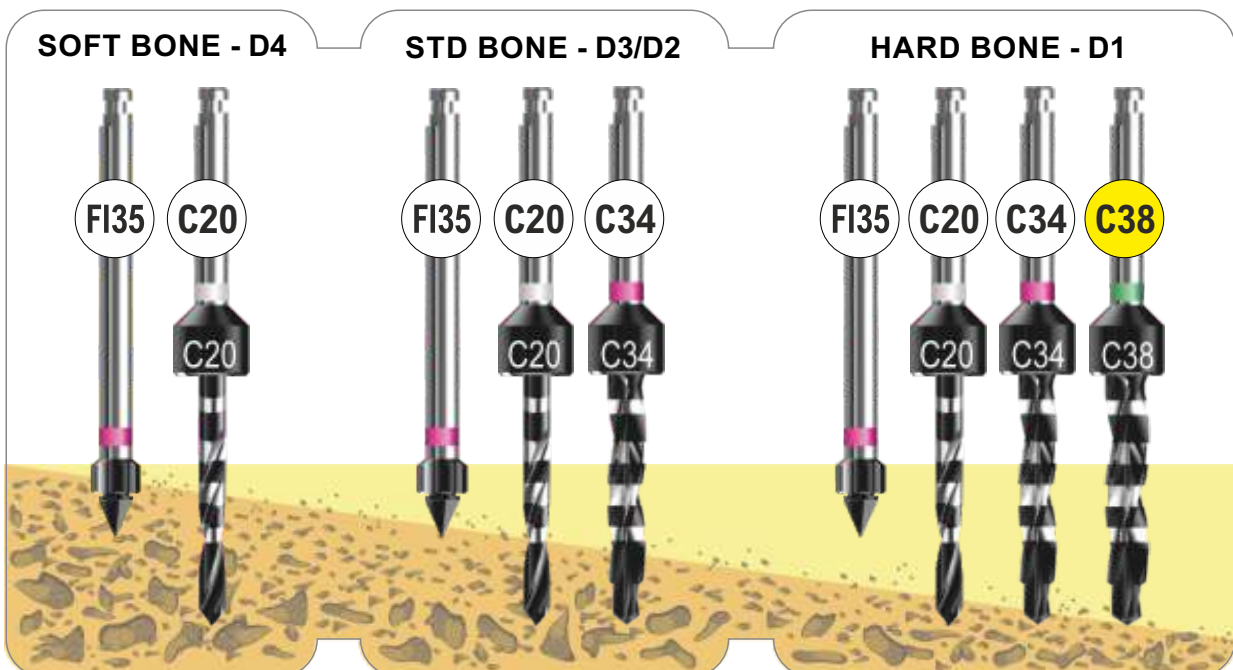
**NARROW S - PLATINUM**  
conical connection 12° - full treatment

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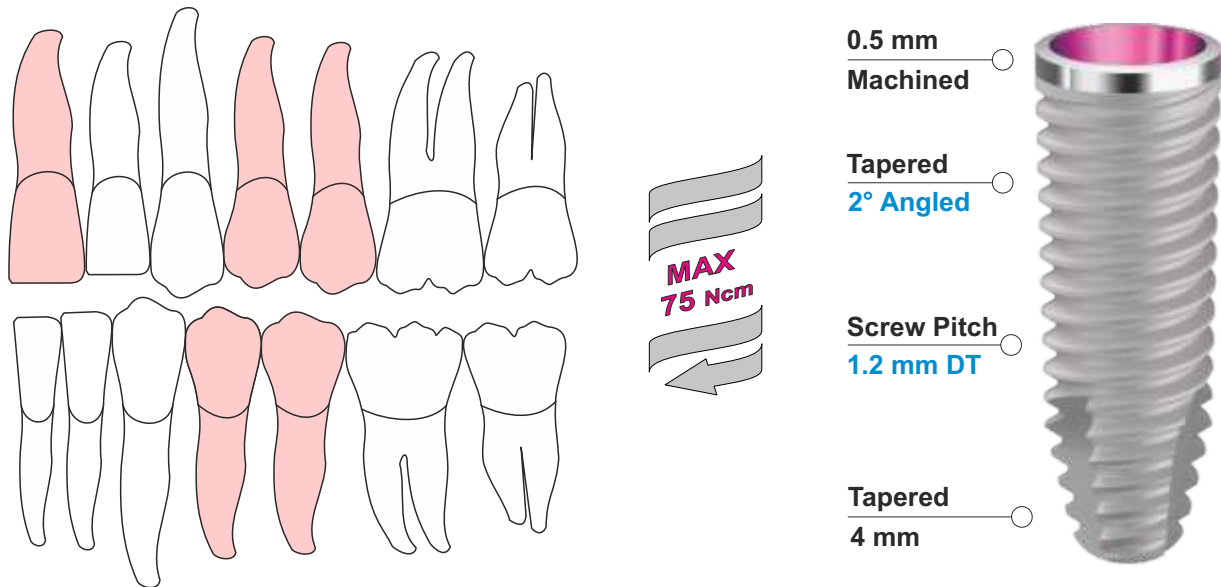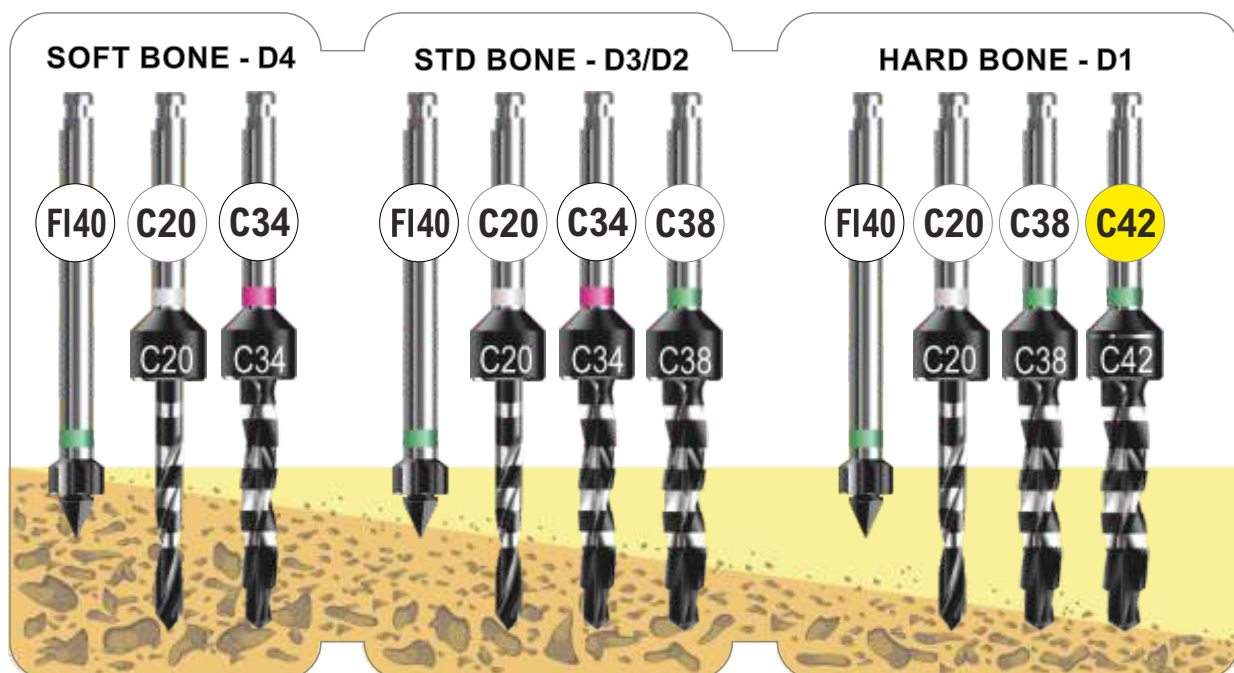Healing Abutment - Titanium Gr. 5 - Anodized

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Laboratory Analog - Titanium Gr. 5 - Anodized

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25Ncm 

## Moncone Titanio Dritto - Titanio Gr. 5

Straight Titanium Abutment - Titanium Gr. 5

**IP 3ST1-P** 0° - Straight - Narrow Gingival Profile H1.5mm

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25Ncm 

## Moncone Titanio Preangolato 15° - Titanio Gr. 5

15° Angled Titanium Abutment - Titanium Gr. 5

**IP 3AT151-P** 15° - Angled - Narrow Gingival Profile H1.5mm

**IP 3AT153-P** 15° - Angled - Narrow Gingival Profile H3.0mm

**IP 3AT154-P** 15° - Angled - Narrow Gingival Profile H4.0mm

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# narrow CCP prosthetics

## Moncone Titanio Preangolato 25° - Titanio Gr. 5

25° Angled Titanium Abutment - Titanium Gr. 5

**IP 3AT251-P** 25° - Angled - Narrow Gingival Profile H1.5mm

**IP 3AT253-P** 25° - Angled - Narrow Gingival Profile H3.0mm

**IP 3AT254-P** 25° - Angled - Narrow Gingival Profile H4.0mm

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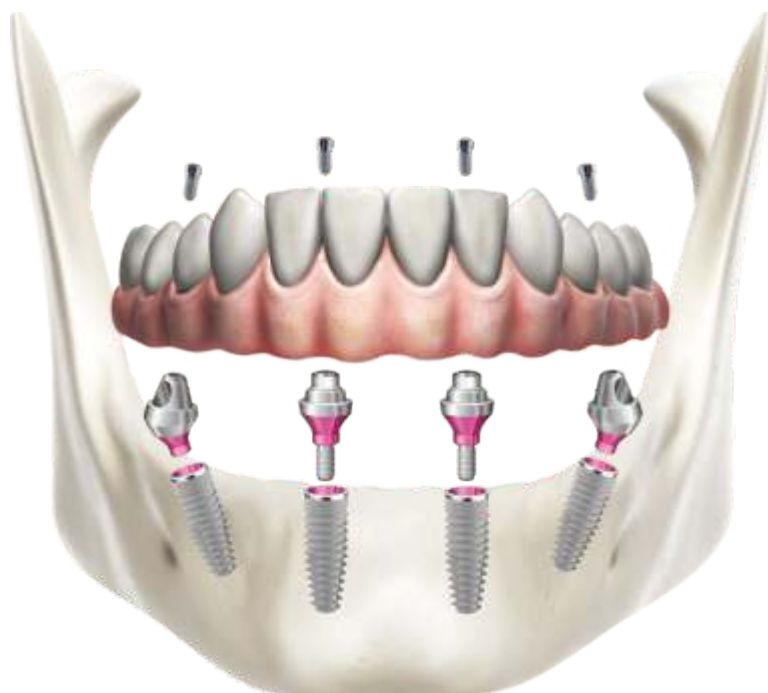
25Ncm

## Pilastro Toronto Diritto - Titanio Gr. 5

Toronto Straight Abutment - Titanium Gr. 5

- IP 3MS1-P H 1.5 mm - Standard Gingival Profile
- IP 3MS3-P H 3.0 mm - Standard Gingival Profile
- IP 3MS4-P H 4.0 mm - Standard Gingival Profile

DCDM

Toronto Straight Abutment Driver

25Ncm

## Pilastro Toronto Angolato 17° - Titanio Gr. 5

Toronto 17° Angled Abutment - Titanium Gr. 5

- IP 3MA172-P 17° - H 2 mm - Standard Gingival Profile
- IP 3MA173-P 17° - H 3 mm - Standard Gingival Profile

IP 3VITMA-P Vite di Ricambio Toronto - Toronto Spare Screw  
DMX 1/2 Standard Torx Driver

25Ncm

## Pilastro Toronto Angolato 30° - Titanio Gr. 5

Toronto 30° Angled Abutment - Titanium Gr. 5

- IP 3MA303-P 30° - H 3 mm - Standard Gingival Profile
- IP 3MA304-P 30° - H 4 mm - Standard Gingival Profile

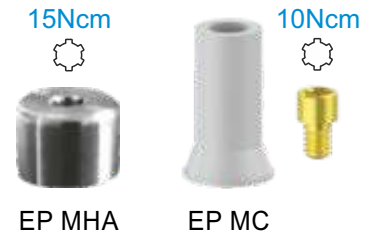
IP 3VITMA-P Vite di Ricambio Toronto - Toronto Spare Screw  
DMX 1/2 Standard Torx Driver

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### Toronto Castable Abutment & Healing Abutment

- EP MC** Rotating Castable Abutment
- EP MHA** H 4 mm - Standard Gingival Profile Titanium Gr5
- EP MHA6** H 6 mm - Large Gingival Profile Titanium Gr5
- EP MV** Vite di Ricambio Toronto - Toronto Spare Screw
- DMX 1/2** Standard Torx Driver

## Componenti Protesiche Toronto - Titanio Gr. 5

### Toronto Prosthetics - Titanium Gr. 5

- EP MI** Rotating - Standard Gingival Profile
- EP MAL2DS** Digital Toronto Analog - Analogo Digitale
- EP MT** Rotating Temporary - Incollaggio Passivo
- EP MTS** Rotating Welding Temporary - Saldatura Intraorale
- EP MVL** Vite di Ricambio Lunga - Long Spare Screw
- DMX 1/2** Standard Torx Driver

## Transfer Toronto & Link da Incollaggio - Titanio Gr. 5

### Toronto Scan Abutment & Link - Titanium Gr. 5

- EP MSA2 DS** Toronto Rotating Scan Abutment
- EP MT DS** Toronto Rotating - Bonding Technique Link
- EP MV** Vite di Ricambio Toronto - Toronto Spare Screw
- DMX 1/2** Standard Torx Driver

## Digital Solutions

## Componenti Digitali CCP - Titanio Gr. 5

### CCP Digital Components - Titanium Gr. 5

- ACT SB35K** Antirotational Scan Abutment
- ACT LKN35K** Antirotational Bonding Technique Link
- ACT LKR35K** Rotating Bonding Technique Link
- ACT AN3D35K** CAD Laboratory Analog
- ACT VP16** Vite di Ricambio - Spare Screw
- DMX 1/2** Standard Torx Driver

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

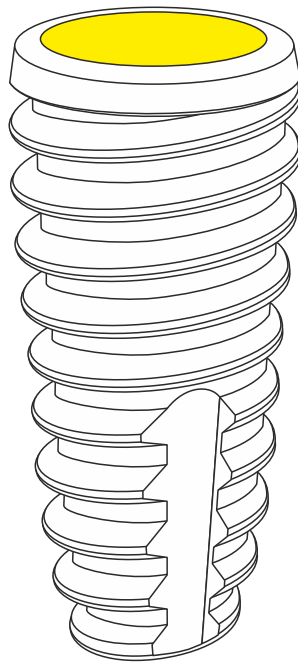
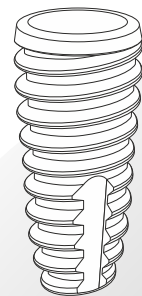
# IK 42-P

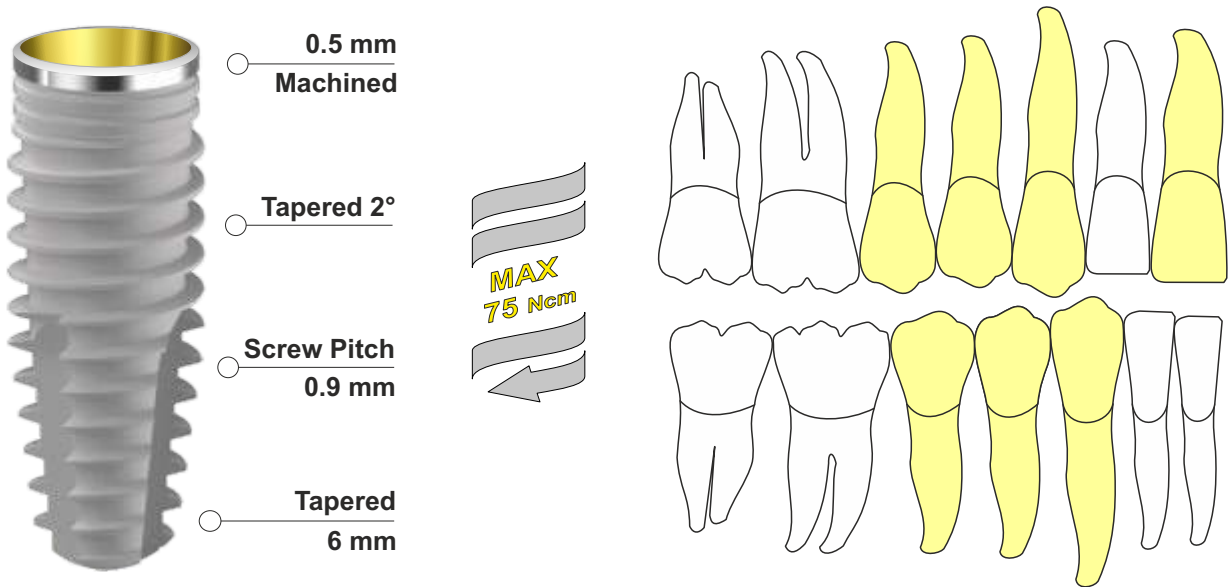REGULAR K - PLATINUM  
conical connection 12° - full treatment

## TAPERED SHAPE MOUNTERLESS




CODE	IMPLANT	LENGTH	NECK	APEX
IK 4207 P	Ø 4.2	7.0 mm	Ø 4.0	Ø 2.2
IK 4208 P	Ø 4.2	8.5 mm	Ø 4.0	Ø 2.2
IK 4210 P	Ø 4.2	10 mm	Ø 4.0	Ø 2.2
IK 4211 P	Ø 4.2	11.5 mm	Ø 4.0	Ø 2.2
IK 4213 P	Ø 4.2	13 mm	Ø 4.0	Ø 2.2
IK 4215 P	Ø 4.2	15 mm	Ø 4.0	Ø 2.2

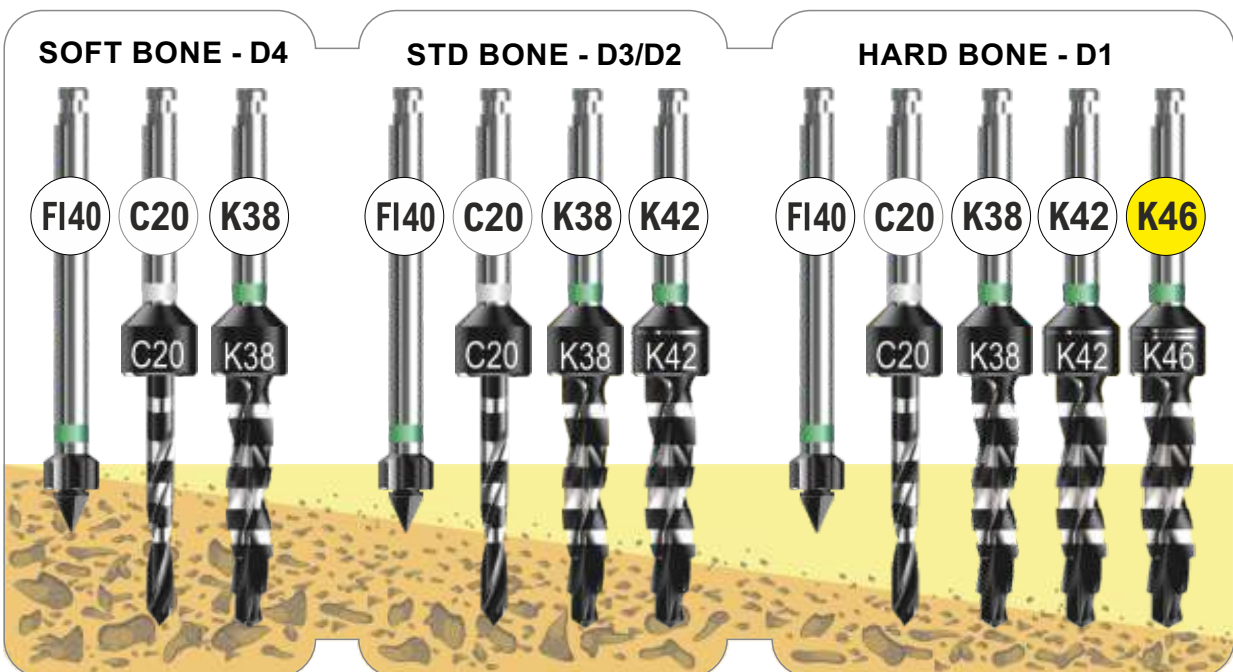
**REGULAR K - PLATINUM**  
conical connection 12° - full treatment

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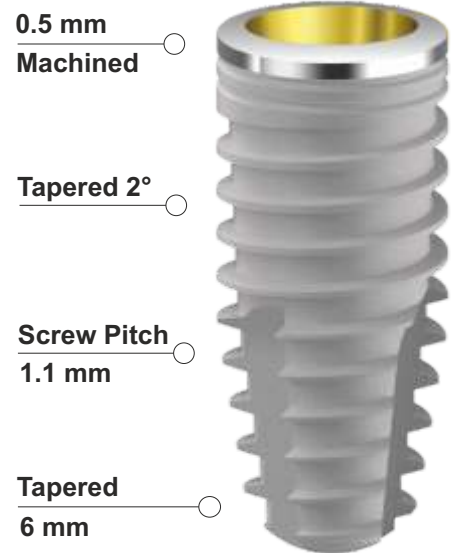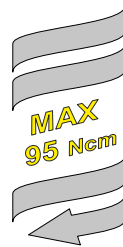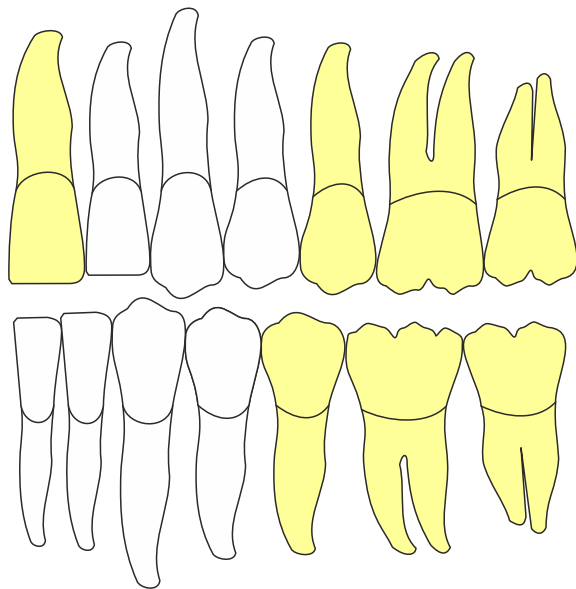
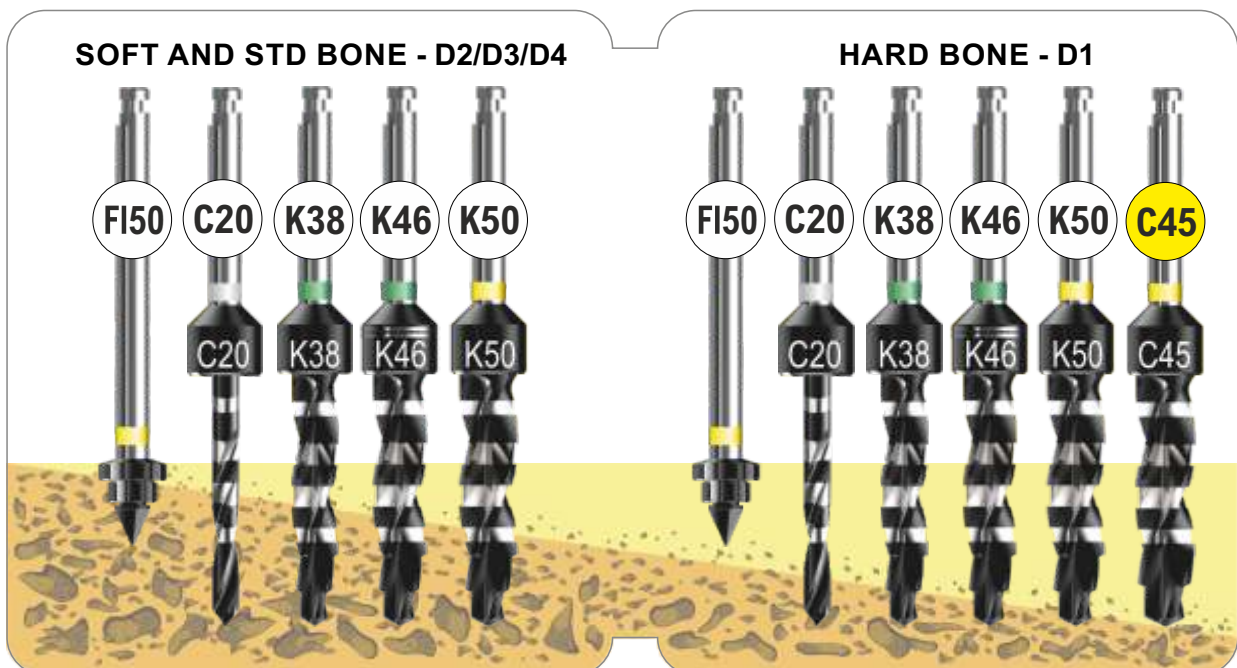
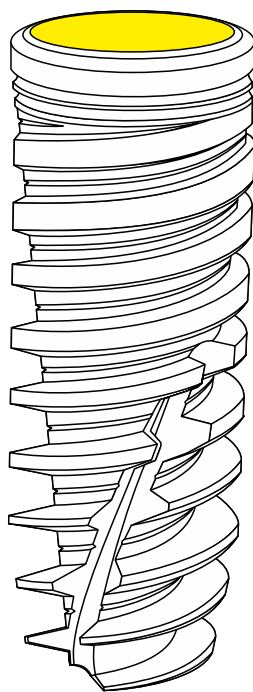
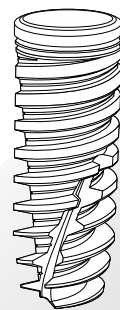
# IA 42-P

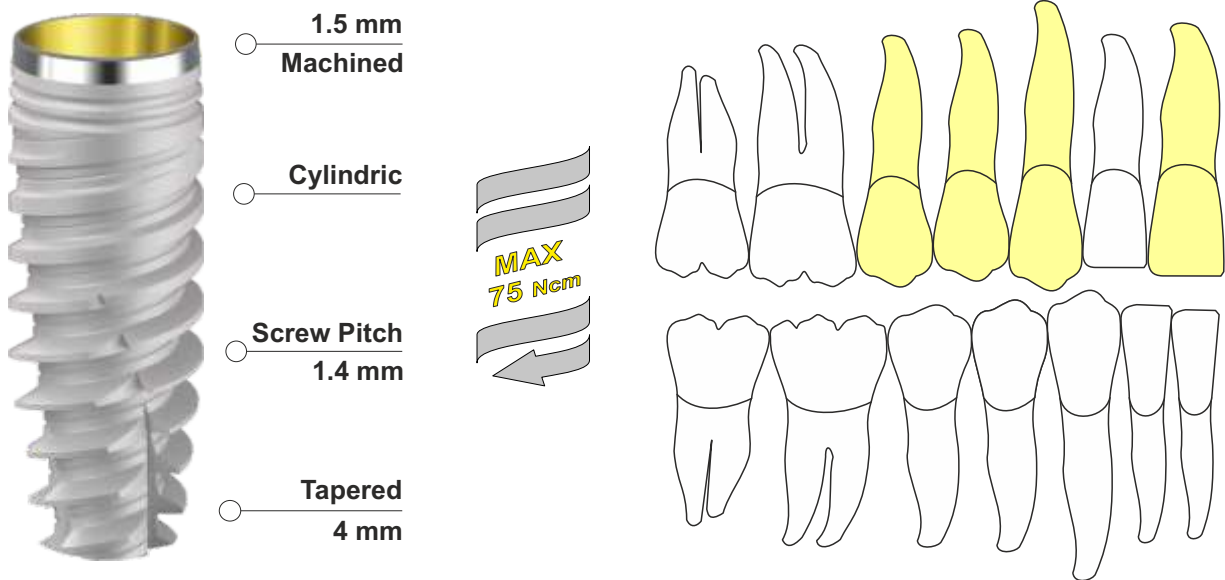REGULAR A - PLATINUM  
conical connection 12° - full treatment

## ACTIVE BLADE SHAPE MOUNTERLESS



CODE	IMPLANT	LENGTH	NECK	APEX
IA 4208 P	Ø 4.2	8.5 mm	Ø 4.0	Ø 2.2
IA 4210 P	Ø 4.2	10 mm	Ø 4.0	Ø 2.2
IA 4211 P	Ø 4.2	11.5 mm	Ø 4.0	Ø 2.2
IA 4213 P	Ø 4.2	13 mm	Ø 4.0	Ø 2.2
IA 4215 P	Ø 4.2	15 mm	Ø 4.0	Ø 2.2

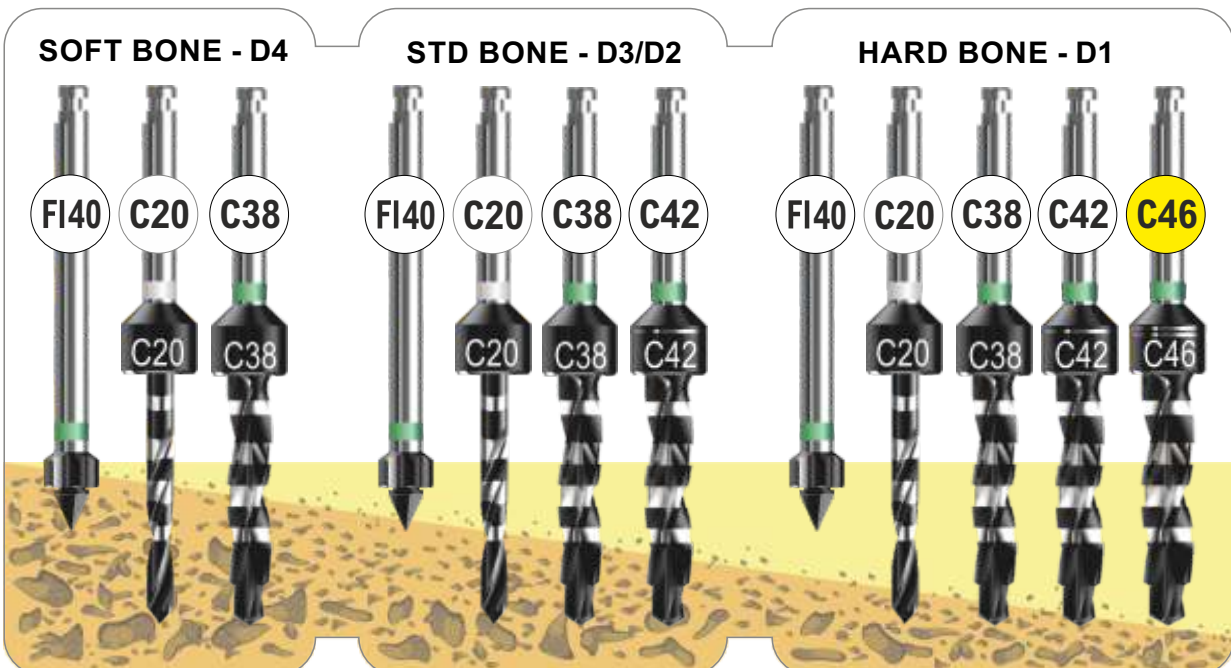
**REGULAR A - PLATINUM**  
conical connection 12° - full treatment

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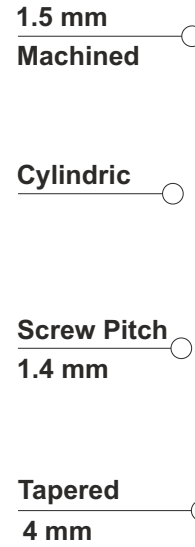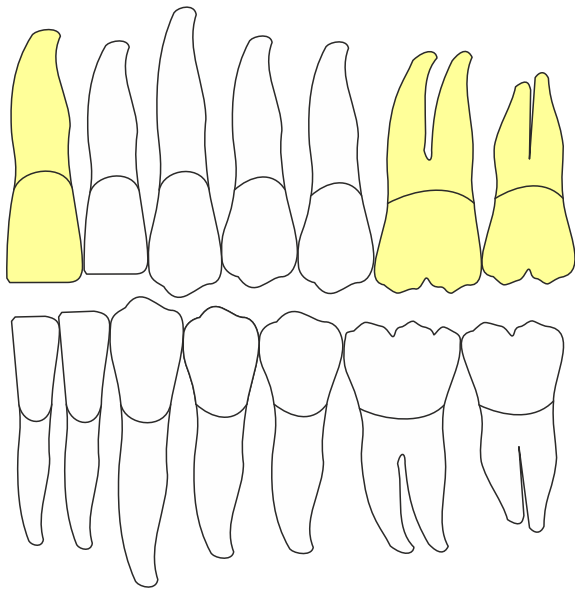
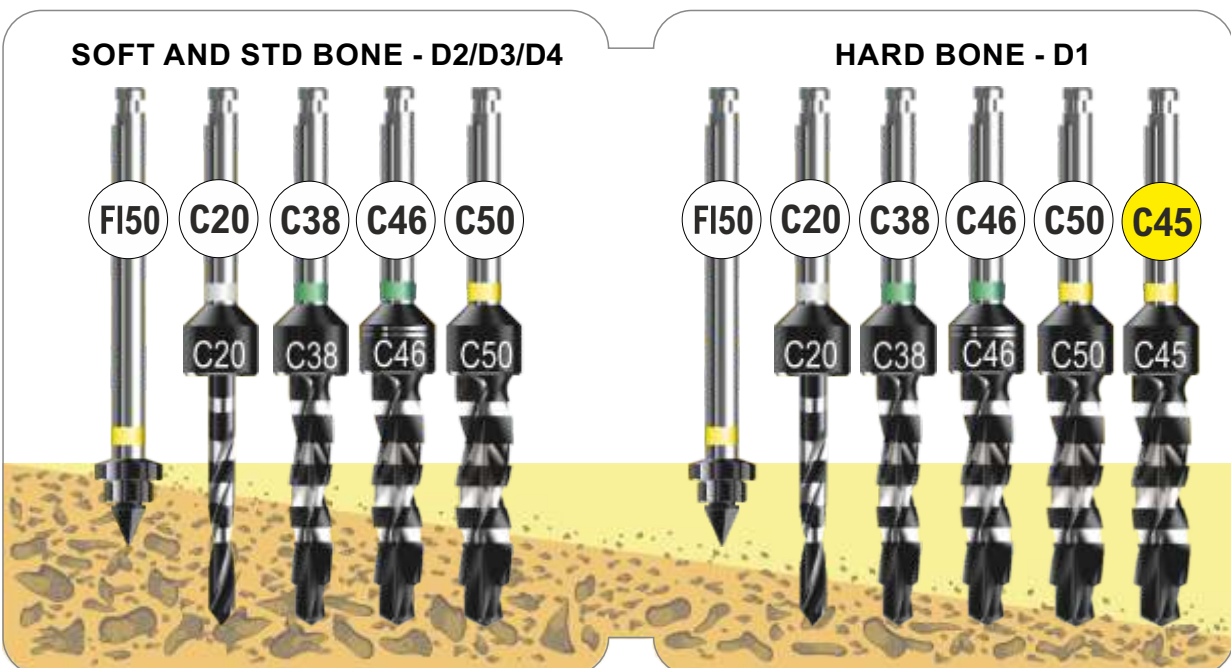
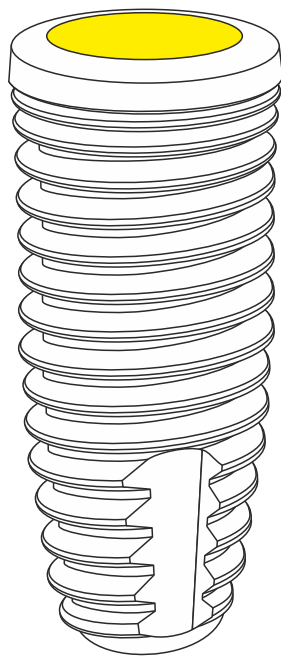
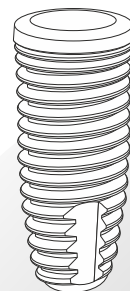
# ES 42-P

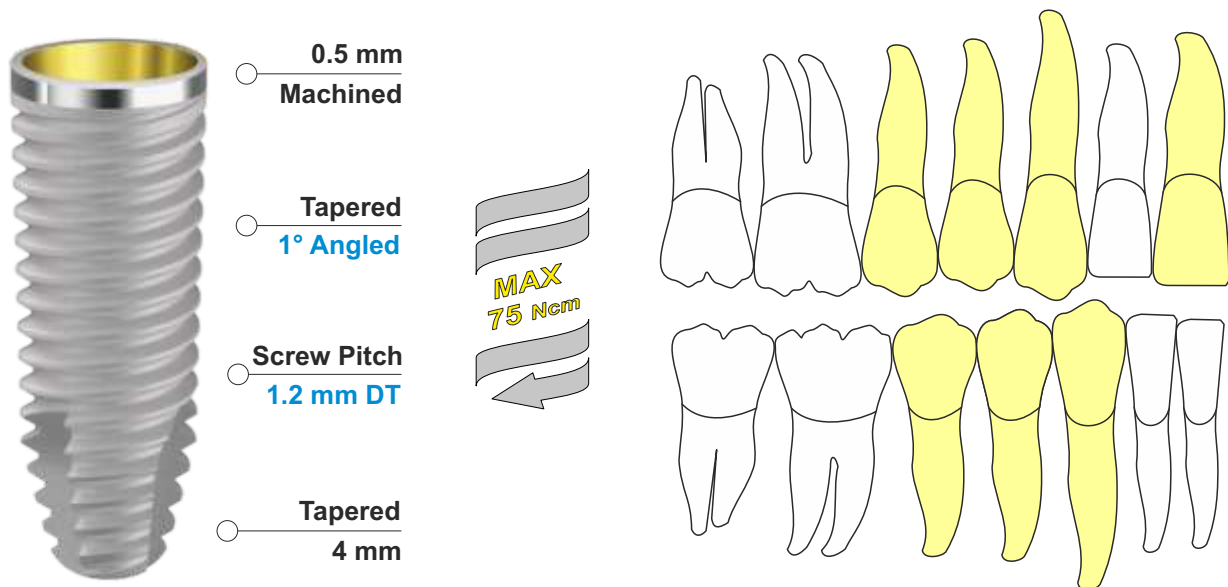REGULAR S - PLATINUM  
conical connection 12° - full treatment

## SPEEDY SHAPE MOUNTERLESS


CODE	IMPLANT	LENGTH	NECK	APEX
ES 4208 P	Ø 4.2	8.5 mm	Ø 4.0	Ø 2.2
ES 4210 P	Ø 4.2	10 mm	Ø 4.0	Ø 2.2
ES 4211 P	Ø 4.2	11.5 mm	Ø 4.0	Ø 2.2
ES 4213 P	Ø 4.2	13 mm	Ø 4.0	Ø 2.2
ES 4215 P	Ø 4.2	15 mm	Ø 4.0	Ø 2.2

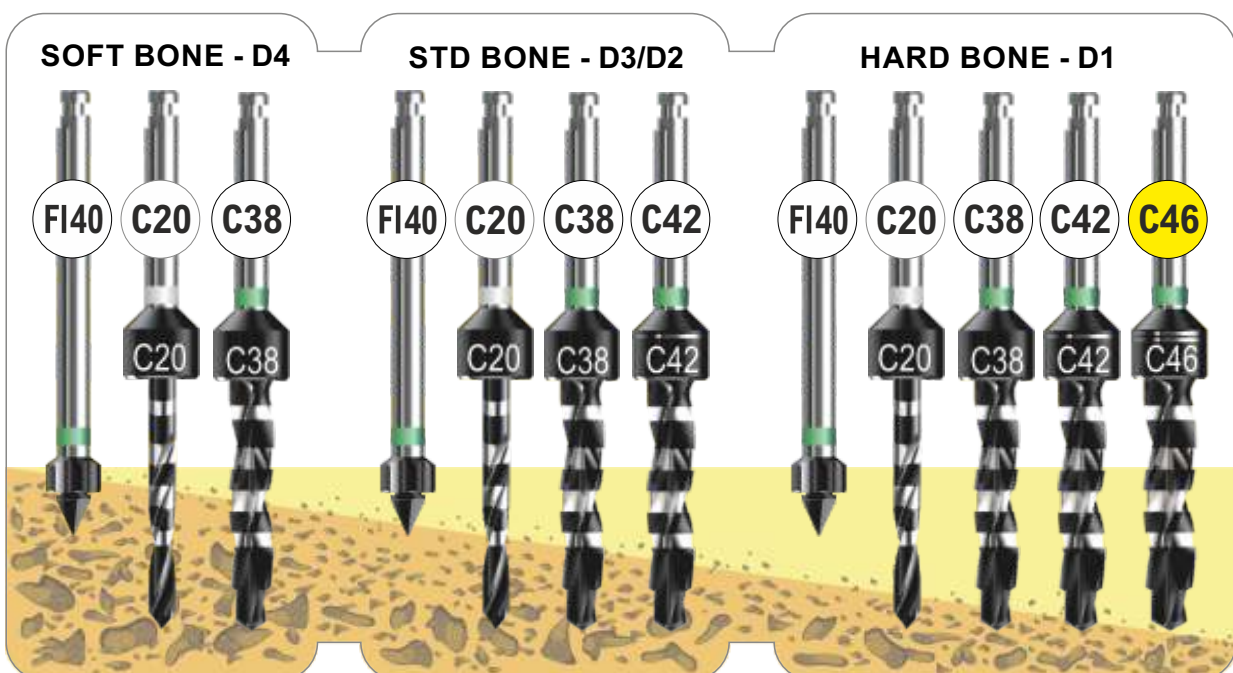
**REGULAR S - PLATINUM**  
conical connection 12° - full treatment

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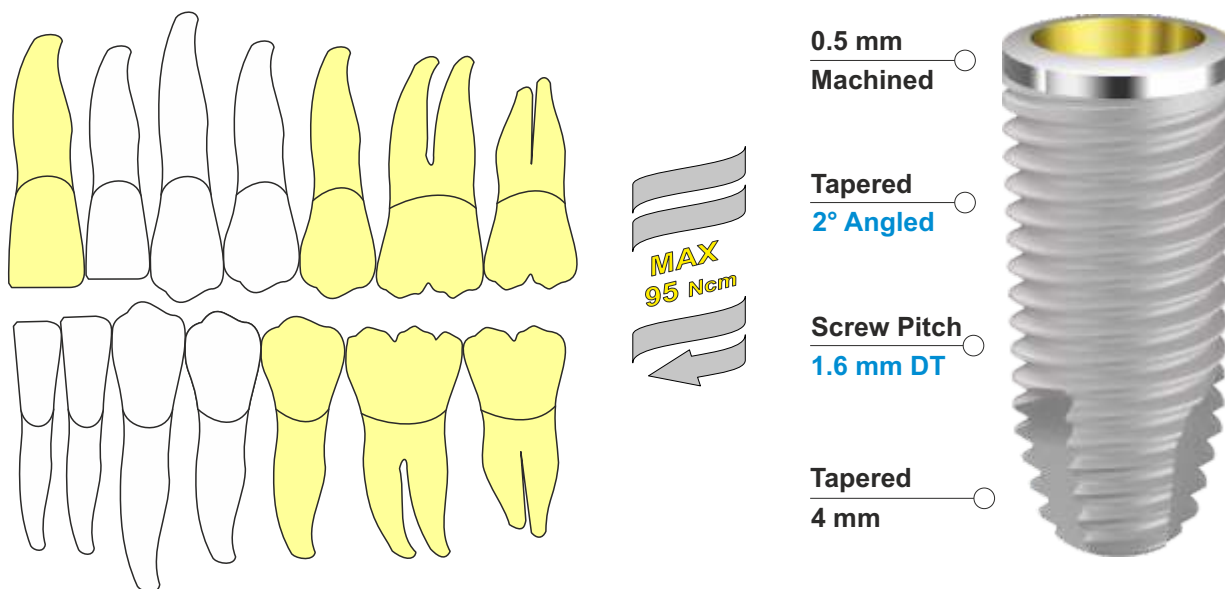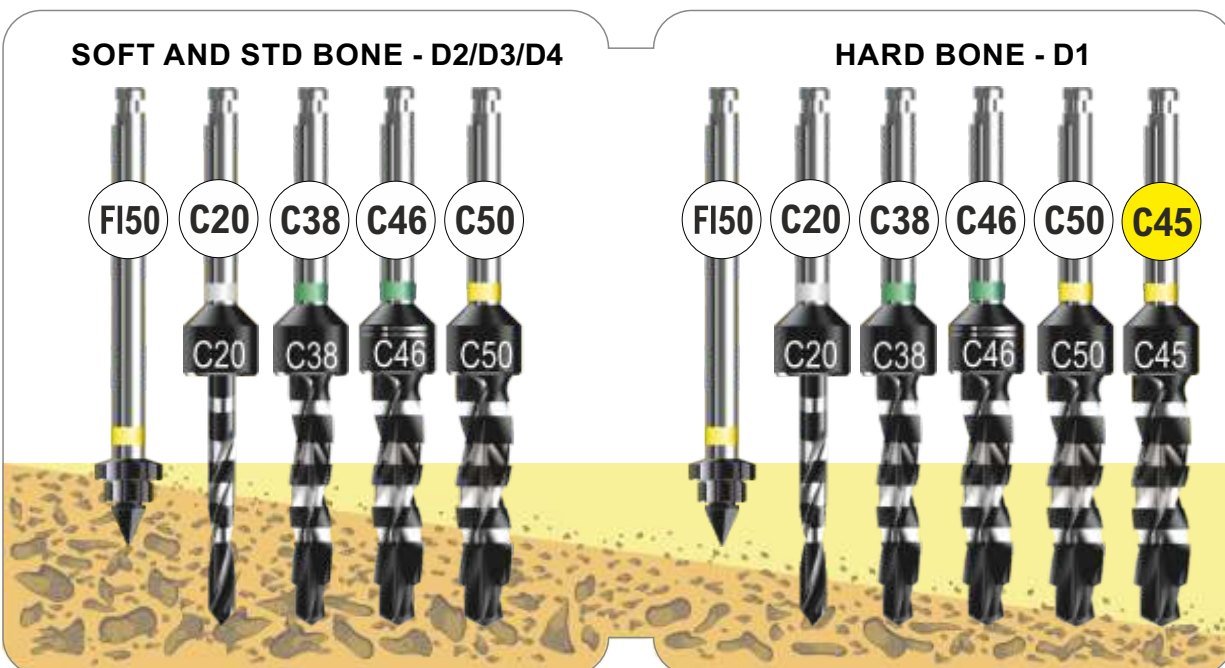**Healing Abutment** - Titanium Gr. 5 - Anodized

**IP 4HA3-P** H 3 mm - Regular Gingival Profile

**IP 4HA4-P** H 4 mm - Regular Gingival Profile

**IP 4HA5-P** H 5 mm - Regular Gingival Profile

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**Laboratory Analog** - Titanium Gr. 5 - Anodized

**IP 4AL-P** H 12 mm - Universal Shape

32Ncm 

**Moncone UCLA Cobalto Cromo** - CoCr

**UCLA Abutment Cobalt Chrome** - CoCr

**IP 4CCA-P** Antirotation Hex - Regular Gingival Profile H1,5mm

**IP 4CCR-P** Rotating - Regular Gingival Profile H1,5mm

**SP VIT40** Vite di Ricambio - Spare Screw

**DMX 1/2** Standard Torx Driver

32Ncm 

**Moncone Titanio Dritto** - Titanio Gr. 5

**Straight Titanium Abutment** - Titanium Gr. 5

**IP 4ST1-P** 0° - Straight - Regular Gingival Profile H1.5mm

**IP 4ST3-P** 0° - Straight - Regular Gingival Profile H3.0mm

**IP 4ST4-P** 0° - Straight - Regular Gingival Profile H4.0mm

**SP VIT40** Vite di Ricambio - Spare Screw

**DMX 1/2** Standard Torx Driver

32Ncm 

**Moncone Titanio Preangolato 15°** - Titanio Gr. 5

**15° Angled Titanium Abutment** - Titanium Gr. 5

**IP 4AT151-P** 15° - Angled - Regular Gingival Profile H1.5mm

**IP 4AT153-P** 15° - Angled - Regular Gingival Profile H3.0mm

**IP 4AT154-P** 15° - Angled - Regular Gingival Profile H4.0mm

**SP VIT40** Vite di Ricambio - Spare Screw

**DMX 1/2** Standard Torx Driver

# regular CCP prosthetics

## Moncone Titanio Preangolato 25° - Titanio Gr. 5

25° Angled Titanium Abutment - Titanium Gr. 5

**IP 4AT251-P** 25° - Angled - Regular Gingival Profile H1.5mm

**IP 4AT253-P** 25° - Angled - Regular Gingival Profile H3.0mm

**IP 4AT254-P** 25° - Angled - Regular Gingival Profile H4.0mm

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Antirotation Hex Temporary Abutment - Ti Gr. 5

**IP 4TA1-P** Antirotation Hex - Regular Gingival Profile H 1.5mm

**IP 4TA3-P** Antirotation Hex - Regular Gingival Profile H 3.0mm

**IP 4TR1-P** Rotating - Regular Gingival Profile H 1.5mm

**IP 4TR3-P** Rotating - Regular Gingival Profile H 3.0mm

**DMX 1/2** Standard Torx Driver

## Pilastro per Overdenture - Titanio Gr. 5 - PVD TiN

Overdenture Abutment - Titanium Gr. 5 - PVD TiN

**IP 4BA1-P** H 1 mm - Regular Gingival Profile - NORMO

**IP 4BA2-P** H 2 mm - Regular Gingival Profile - NORMO

**IP 4BA3-P** H 3 mm - Regular Gingival Profile - NORMO

**IP 4BA4-P** H 4 mm - Regular Gingival Profile - NORMO

**DMX 1/2** Standard Torx Driver

## Kit Pilastro Equator per Overdenture - Titanio Gr. 5 - PVD

Overdenture Equator Abutment Kit - Titanium Gr. 5 - PVD

**IP 4EQ1-P** H 1 mm - Regular Gingival Profile

**IP 4EQ2-P** H 2 mm - Regular Gingival Profile

**IP 4EQ3-P** H 3 mm - Regular Gingival Profile

**IP 4EQ5-P** H 5 mm - Regular Gingival Profile

**DM EQ** Squared Equator Driver

## Transfer per impronta Pick Up - Titanio Gr. 5

Pick Up Impression Coping - Titanium Gr. 5

**IP 4IA1-P** Regular Gingival Profile - H1.5mm

**IP 4IA3-P** Regular Gingival Profile - H3.0mm

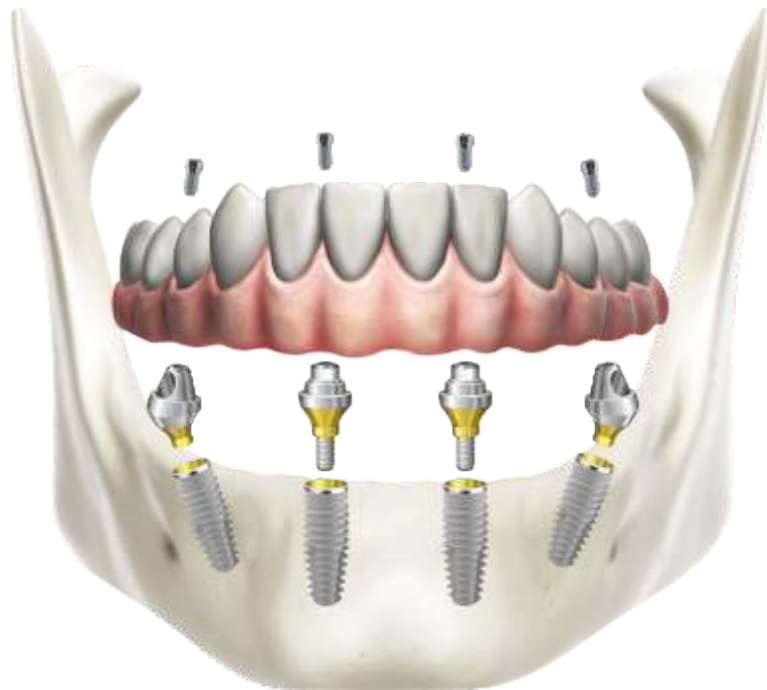
**IP 4IA4-P** Regular Gingival Profile - H4.0mm

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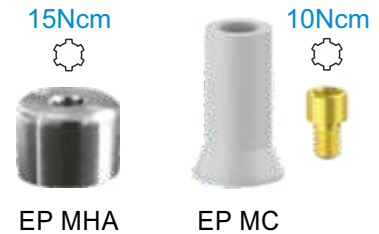
### Toronto Castable Abutment & Healing Abutment

- EP MC** Rotating Castable Abutment
- EP MHA** H 4 mm - Standard Gingival Profile Titanium Gr5
- EP MHA6** H 6 mm - Large Gingival Profile Titanium Gr5
- EP MV** Vite di Ricambio Toronto - Toronto Spare Screw
- DMX 1/2** Standard Torx Driver

## Componenti Protesiche Toronto - Titanio Gr. 5

### Toronto Prosthetics - Titanium Gr. 5

- EP MI** Rotating - Standard Gingival Profile
- EP MAL2DS** Digital Toronto Analog - Analogo Digitale
- EP MT** Rotating Temporary - Incollaggio Passivo
- EP MTS** Rotating Welding Temporary - Saldatura Intraorale
- EP MVL** Vite di Ricambio Lunga - Long Spare Screw
- DMX 1/2** Standard Torx Driver

## Transfer Toronto & Link da Incollaggio - Titanio Gr. 5

### Toronto Scan Abutment & Link - Titanium Gr. 5

- EP MSA2 DS** Toronto Rotating Scan Abutment
- EP MT DS** Toronto Rotating - Bonding Technique Link
- EP MV** Vite di Ricambio Toronto - Toronto Spare Screw
- DMX 1/2** Standard Torx Driver

## Digital Solutions

## Componenti Digitali CCP - Titanio Gr. 5

### CCP Digital Components - Titanium Gr. 5

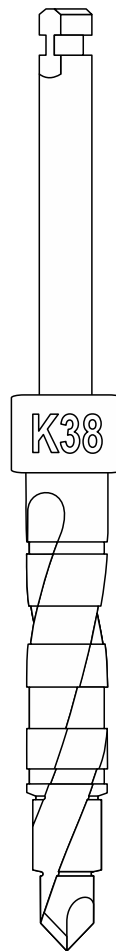
- ACT SB43K** Antirotational Scan Abutment
- ACT LKN43K** Antirotational Bonding Technique Link
- ACT LKR43K** Rotating Bonding Technique Link
- ACT AN3D43K** CAD Laboratory Analog
- ACT VP** Vite di Ricambio - Spare Screw
- DMX 1/2** Standard Torx Driver

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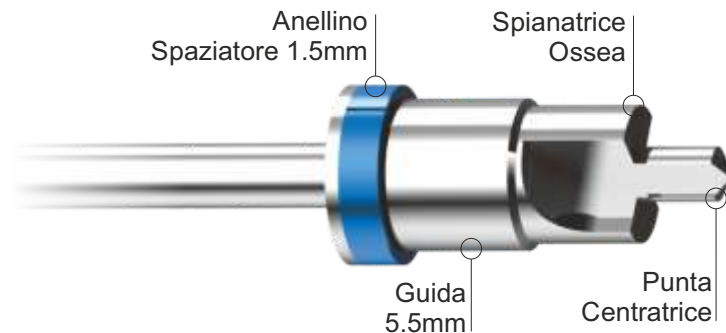
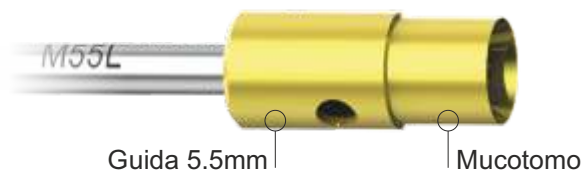
# guided surgical drills

## FG C - CORTICAL DRILLS

- FGR C35 S** Corticall Drill - Ø 3.5 mm - ● Small
- FGR C40 S** Corticall Drill - Ø 4.0 mm - ● Small
- FGR C35 L** Corticall Drill - Ø 3.5 mm - ● Large
- FGR C40 L** Corticall Drill - Ø 4.0 mm - ● Large
- FGR C50 L** Corticall Drill - Ø 5.0 mm - ● Large

## FG 20 S - FIRST DRILLS ● Small Driver

- FGR 2008 S** First Drill - Ø 2.0 mm - Long 8,5 mm
- FGR 2010 S** First Drill - Ø 2.0 mm - Long 10 mm
- FGR 2011 S** First Drill - Ø 2.0 mm - Long 11,5 mm
- FGR 2013 S** First Drill - Ø 2.0 mm - Long 13 mm
- FGR 2015 S** First Drill - Ø 2.0 mm - Long 15 mm

## FG 20 L - FIRST DRILLS ● Large Driver

- FG 2008 L** First Drill - Ø 2.0 mm - Long 8,5 mm
- FGR 2010 L** First Drill - Ø 2.0 mm - Long 10 mm
- FGR 2011 L** First Drill - Ø 2.0 mm - Long 11,5 mm
- FGR 2013 L** First Drill - Ø 2.0 mm - Long 13 mm
- FGR 2015 L** First Drill - Ø 2.0 mm - Long 15 mm

# guided surgical drills

## FG 34 L - 3.4 IMPLANT DRILLS ● Small Driver

- FGR 3408 S** 3.4 Implant Drill - Ø 2.9 mm - Long 8,5 mm
- FGR 3410 S** 3.4 Implant Drill - Ø 2.9 mm - Long 10 mm
- FGR 3411 S** 3.4 Implant Drill - Ø 2.9 mm - Long 11,5 mm
- FGR 3413 S** 3.4 Implant Drill - Ø 2.9 mm - Long 13 mm
- FGR 3415 S** 3.4 Implant Drill - Ø 2.9 mm - Long 15 mm

## FG 34 L - 3.4 IMPLANT DRILLS ● Large Driver

- FGR 3408 L** 3.4 Implant Drill - Ø 2.9 mm - Long 8,5 mm
- FGR 3410 L** 3.4 Implant Drill - Ø 2.9 mm - Long 10 mm
- FGR 3411 L** 3.4 Implant Drill - Ø 2.9 mm - Long 11,5 mm
- FGR 3413 L** 3.4 Implant Drill - Ø 2.9 mm - Long 13 mm
- FGR 3415 L** 3.4 Implant Drill - Ø 2.9 mm - Long 15 mm

## FG 38 S - 3.8 IMPLANT DRILLS ● Small Driver

- FGR 3808 S** 3.8 Implant Drill - Ø 3.3 mm - Long 8,5 mm
- FGR 3810 S** 3.8 Implant Drill - Ø 3.3 mm - Long 10 mm
- FGR 3811 S** 3.8 Implant Drill - Ø 3.3 mm - Long 11,5 mm
- FGR 3813 S** 3.8 Implant Drill - Ø 3.3 mm - Long 13 mm
- FGR 3815 S** 3.8 Implant Drill - Ø 3.3 mm - Long 15 mm

## FG 38 L - 3.8 IMPLANT DRILLS ● Large Driver

- FGR 3808 L** 3.8 Implant Drill - Ø 3.3 mm - Long 8,5 mm
- FGR 3810 L** 3.8 Implant Drill - Ø 3.3 mm - Long 10 mm
- FGR 3811 L** 3.8 Implant Drill - Ø 3.3 mm - Long 11,5 mm
- FGR 3813 L** 3.8 Implant Drill - Ø 3.3 mm - Long 13 mm
- FGR 3815 L** 3.8 Implant Drill - Ø 3.3 mm - Long 15 mm

# guided surgical drills

## FG 42 S - 4.2 IMPLANT DRILLS ● Small Driver

<b>FGR 4208 S</b>	4.2 Implant Drill - Ø 3.6 mm - Long 8,5 mm
<b>FGR 4210 S</b>	4.2 Implant Drill - Ø 3.6 mm - Long 10 mm
<b>FGR 4211 S</b>	4.2 Implant Drill - Ø 3.6 mm - Long 11,5 mm
<b>FGR 4213 S</b>	4.2 Implant Drill - Ø 3.6 mm - Long 13 mm
<b>FGR 4215 S</b>	4.2 Implant Drill - Ø 3.6 mm - Long 15 mm

## FG 42 L - 4.2 IMPLANT DRILLS ● Large Driver

<b>FGR 4208 L</b>	4.2 Implant Drill - Ø 3.6 mm - Long 8,5 mm
<b>FGR 4210 L</b>	4.2 Implant Drill - Ø 3.6 mm - Long 10 mm
<b>FGR 4211 L</b>	4.2 Implant Drill - Ø 3.6 mm - Long 11,5 mm
<b>FGR 4213 L</b>	4.2 Implant Drill - Ø 3.6 mm - Long 13 mm
<b>FGR 4215 L</b>	4.2 Implant Drill - Ø 3.6 mm - Long 15 mm

## FG 46 L - 4.6 IMPLANT DRILLS ● Large Driver

<b>FGR 4608 L</b>	4.6 Implant Drill - Ø 3.9 mm - Long 8,5 mm
<b>FGR 4610 L</b>	4.6 Implant Drill - Ø 3.9 mm - Long 10 mm
<b>FGR 4611 L</b>	4.6 Implant Drill - Ø 3.9 mm - Long 11,5 mm
<b>FGR 4613 L</b>	4.6 Implant Drill - Ø 3.9 mm - Long 13 mm
<b>FGR 4615 L</b>	4.6 Implant Drill - Ø 3.9 mm - Long 15 mm

## FG 50 L - 5.0 IMPLANT DRILLS ● Large Driver

<b>FGR 5008 L</b>	5.0 Implant Drill - Ø 4.3 mm - Long 8,5 mm
<b>FGR 5010 L</b>	5.0 Implant Drill - Ø 4.3 mm - Long 10 mm
<b>FGR 5011 L</b>	5.0 Implant Drill - Ø 4.3 mm - Long 11,5 mm
<b>FGR 5013 L</b>	5.0 Implant Drill - Ø 4.3 mm - Long 13 mm

# guided surgical drills

## GUIDED MOUNTERS

- IP4MG 4S** Mounter Guidata - Internal Hex ● Small Drive
- IP4MG 4L** Mounter Guidata - Internal Hex ● Large Drive
  
- SP3MG 3S** Mounter Guidata - External Hex ● Small Drive
- SP4MG 4S** Mounter Guidata - External Hex ● Small Drive
- SP4MG 4L** Mounter Guidata - External Hex ● Large Drive
  
- IP3MG 3S-P** Mounter Guidata - Platinum CC ● Small Drive
- IP3MG 3L-P** Mounter Guidata - Platinum CC ● Large Drive
- IP4MG 4L-P** Mounter Guidata - Platinum CC ● Large Drive

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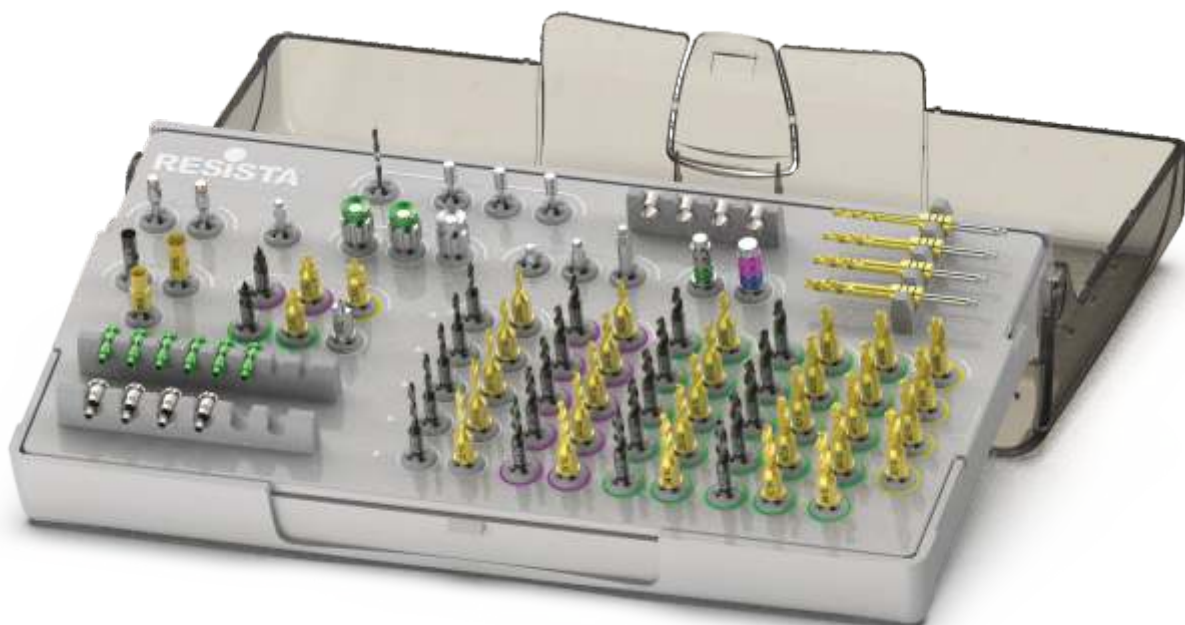
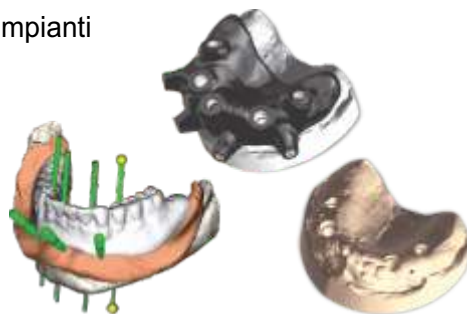
# computer guided surgery

**MASCHERINA CHIRURGICA** - Mascherina on demand multi-impianti

**SURGICAL GUIDE** - Multi-implants on demand surgical guide

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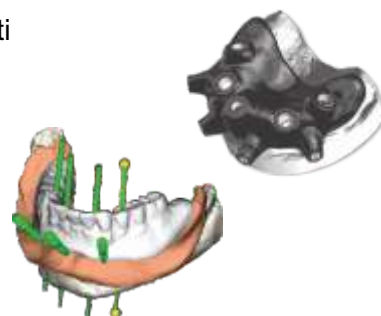
**MASTER MODEL** - Pre-drilled stereolithographed model for analog

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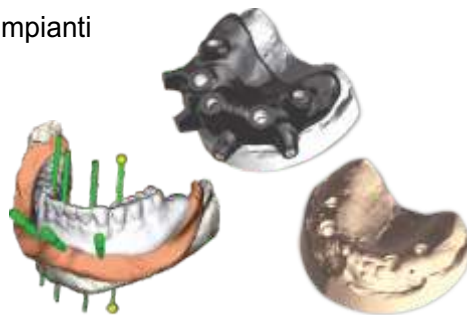
**IMMEDIATE TEMPORARY PROSTHESIS** - Drilled PMMA + Milled Titanium Framework

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### Kit OT Equator Conical Connection - Abutment + Box + Caps

- IP3EQ1-P Kit OT Equator h1 mm Conical C. - NP ●
- IP3EQ2-P Kit OT Equator h2 mm Conical C. - NP ●
- IP3EQ3-P Kit OT Equator h3 mm Conical C. - NP ●
- IP3EQ4-P Kit OT Equator h4 mm Conical C. - NP ●
- IP3EQ5-P Kit OT Equator h5 mm Conical C. - NP ●
- IP3EQ6-P Kit OT Equator h6 mm Conical C. - NP ●

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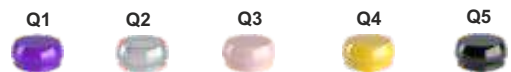
### Kit OT Equator Conical Connection - Abutment + Box + Caps


- IP4EQ1-P Kit OT Equator h1 mm Conical C. - RP ●
- IP4EQ2-P Kit OT Equator h2 mm Conical C. - RP ●
- IP4EQ3-P Kit OT Equator h3 mm Conical C. - RP ●
- IP4EQ4-P Kit OT Equator h4 mm Conical C. - RP ●
- IP4EQ5-P Kit OT Equator h5 mm Conical C. - RP ●
- IP4EQ6-P Kit OT Equator h6 mm Conical C. - RP ●

- TF Q1 Teflon Viola Forte
- TF Q2 Teflon Bianco Normale
- TF Q3 Teflon Rosa Morbida
- TF Q4 Teflon Giallo Extra
- TF Q5 Teflon Nero da Laboratorio

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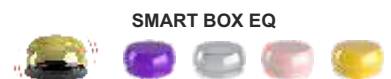P

- AC P3S** Short - Conical Grip Driver **Platinum**
- AC P3M** Medium - Conical Grip Driver **Platinum**
- AC P3L** Long - Conical Grip Driver **Platinum**

- AC P4S** Short - Conical Grip Driver **Platinum**
- AC P4M** Medium - Conical Grip Driver **Platinum**
- AC P4L** Long - Conical Grip Driver **Platinum**

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P

- PC P3S** Short - Conical Grip Driver **Platinum**
- PC P3M** Medium - Conical Grip Driver **Platinum**
- PC P3L** Long - Conical Grip Driver **Platinum**

- PC P4S** Short - Conical Grip Driver **Platinum**
- PC P4M** Medium - Conical Grip Driver **Platinum**
- PC P4L** Long - Conical Grip Driver **Platinum**

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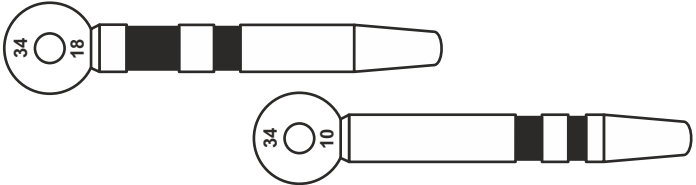
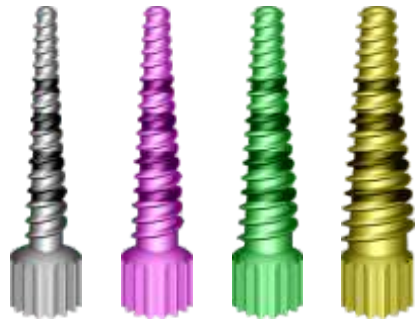
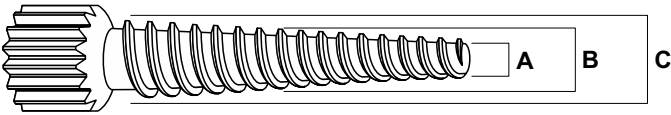
**EXP** 4 Progressive diameters  
**KIT 005** Expander Surgical Kit

# instruments and tools

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Manual Driver - Stainless Steel

**CRND** Cricchetto non Dinamometrico - No Torque Control Ratchet

**CRD2** Cricchetto Dinamometrico 10/70 Ncm - Torque Control

CRD2-G

**CRD2-G** Ghiera Cricchetto CRD2 con Attacco PC - CRD2 Ratchet Wheel with PC Connection

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
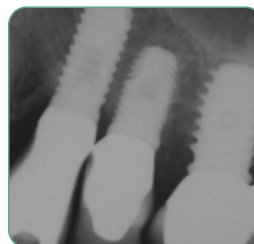
**KIT 007** Implant Extractor Kit - 2 Extractors + 1 Box + CRND

## Siringhe per Anestesia - Inox

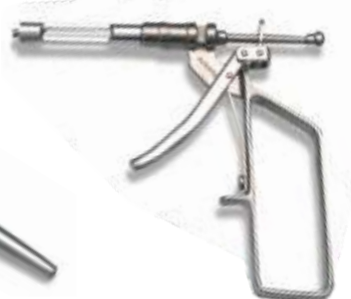Anesthesia Syringes - Stainless Steel

PERIPEN

PERIPRESS

**22** PERIPRESS - Siringa Inox 1.8cc - Set 2 Portafiala

**32** PERIPRESS - Siringa Standard 1.8cc - Set 2 Portafiala

**118** PERIPEN - Siringa Inox 1.8cc - Set 2 Portafiala AM/EU

**Ing. C. A. Issoglio & C. S.r.l.**  
Via F.lli Di Dio, 68 - 28887 Omegna (VB) - ITALIA  
Tel. +39.0323.828.004 - Fax +39.0323.828.914  
[www.resista.it](http://www.resista.it) - [info@resista.it](mailto:info@resista.it)

REV 03 - 31/03/2023